

Piranha

Reference Manual - English - Version 5.7A



Welcome to the Piranha

The Piranha is an X-ray Analyser/Multimeter for everybody working with Quality Assurance and Service of X-ray systems.



**INDEPENDENT X-RAY
QUALITY ASSURANCE**

NOTICE

RTI Group AB reserves all rights to make changes in the Piranha and the information in this document without prior notice.

RTI Group AB assumes no responsibility for any errors or consequential damages that may result from the use or misinterpretation of any information contained in this document.

Copyright © 2001-2019 by RTI Group AB. All rights reserved.

Content of this document may not be reproduced for any other purpose than supporting the use of the product without prior permission from RTI Group AB.

Microsoft, Windows, Win32, Windows XP, 2003, Vista, 7, 8, and 10 are either registered trademarks or trademarks of Microsoft Corporation in the United States and/or other countries.

BLUETOOTH is a trademark owned by Bluetooth SIG, Inc., USA.

Contact Information - World-Wide

*RTI Group AB
Flöjelbergsgatan 8 C
SE-431 37 MÖLNDAL
Sweden*

Phone: Int. +46 31 7463600

*E-mail
Sales: sales@rtigroup.com
Support: support@rtigroup.com
Service: service@rtigroup.com*

Web site: <http://www.rtiigroup.com>

Contact Information - United States

*RTI Electronics Inc.
33 Jacksonville Road, Bldg. 1,
Towaco, NJ 07082,
USA*

*Phone: 800-222-7537 (Toll free)
Int. +1-973-439-0242
Fax: Int. +1-973-439-0248*

*E-mail
Sales: sales.us@rtigroup.com
Support: support.us@rtigroup.com
Service: service.us@rtigroup.com*

Web site: <http://www.rtiigroup.com>

Intended Use of the Piranha System

Together with external probes the Piranha System is intended to be used for independent service and quality control, including measurements of kerma, kerma rate, kVp, tube current, exposure time, luminance, illuminance, and dose area product, within limitations stated below.

If installed according to accompanying documents, the product is intended to be used together with all diagnostic X-ray equipment except for:

- therapeutic X-ray sources.
- X-ray equipment with tube potential below 18 kV or above 160 kV.
- X-ray equipment on which the instrument cannot be mounted properly.
- specific types of X-ray equipment listed in the instructions for use or in additional information from the manufacturer.

With the X-ray installation without patient present, the product is intended to be used:

- for assessing the performance of the X-ray equipment.
- for evaluation of examination techniques and procedures.
- for service and maintenance of the X-ray equipment.
- for quality control of the X-ray equipment.
- for educational purposes, authority supervision etc.

The product is intended to be used by hospital physicists, X-ray engineers, manufacturer's service teams, and other professionals with similar tasks and competencies. The operator needs training to be able to use the product as intended. This training can be achieved either by study of the manual, study of the built-in help function in measurement software or, on request, by a course ordered from the manufacturer.

The product is intended to be used inside X-ray rooms ready for clinical use and can safely be left switched on and in any measuring mode in the vicinity of patients.

The product is NOT intended to be used:

- for direct control of diagnostic X-ray equipment performance during irradiation of a patient.
- so that patients or other unqualified persons can change settings of operating parameters during, immediately before, or after measurements.
- for any guidance to diagnosis of patients.

Table of Contents

1. Introduction.....	3
1.1 About this Manual	4
1.2 Introduction to the Piranha	4
1.3 PC Requirements	5
2. Description of the Piranha.....	6
2.1 Indicators and Connectors	7
2.2 Setting Up the Piranha for the First Time	10
2.3 Setting Up the Piranha	10
2.4 Hardware and Specifications	11
2.4.1 Piranha internal detector (Internal detector)	11
2.4.1.1 General	11
2.4.1.2 Power & Communication Specifications	12
2.4.1.3 Specifications, Piranha	12
2.4.1.4 Typical Response, Piranha	20
2.4.1.5 Angular Sensitivity, Piranha	23
2.4.2 Piranha External Probes	25
2.5 Standards and Compliances	29
2.5.1 Waste Electrical and Electronic Equipment (WEEE)	29
2.5.2 Manufacturer's Declaration of Conformity	30
2.5.3 FCC Certification	31
2.6 Maintenance	31
2.6.1 Updating the Piranha Firmware	31
2.6.2 Managing Detector Calibrations	34
2.7 Measurement Settings	35
2.7.1 Settings - Conditions	36
2.7.2 Settings - Piranha	38
2.7.3 Settings - Internal detector	40
2.7.4 Settings - Other Detectors	41
3. Measurement Principles & Theory.....	42
3.1 Overview of Capability for Measurement Modes	43
3.2 Update Modes	43
3.2.1 Using Timed Update Mode	44
3.2.2 Using Free Run Update Mode	44
3.3 Display Messages and Active Messages	45
3.3.1 Active Messages	45
3.3.2 Display Messages	46
3.4 Waveforms and Triggers	47
3.5 Measurement Principle for the Piranha	48
4. Measurements with the Piranha System.....	49
4.1 Introduction	50
4.2 Radiography	50

2 Contents

4.2.1	kVp, Time, Dose, and Dose Rate	50
4.2.2	Dose Measurements w ith Piranha Dose Probe	51
4.2.3	Quick-HVL and Total Filtration	51
4.3	Cine/Pulsed Radiography	51
4.3.1	kVp, Time, Dose, and Dose Rate	51
4.3.2	Pulse Measurements w ith Piranha Dose Probe	51
4.3.3	HVL, Quick-HVL, and Total Filtration	52
4.4	Fluoroscopy and Pulsed Fluoroscopy	52
4.4.1	Image Intensifier Input Dose Rate	53
4.4.2	Pulsed Fluoroscopy	53
4.5	Mammography	54
4.5.1	General	54
4.5.2	Setting Up the Piranha for Mammography	55
4.5.3	kVp, Time, and Dose Measurements with the Internal detector	56
4.5.4	Dose Measurements w ith the Piranha Dose Probe	56
4.5.5	Quick-HVL	57
4.5.6	Mammo Compensations and Corrections	57
4.5.6.1	Corrections for the Compression Paddle	58
4.5.6.2	Normalization	58
4.5.6.3	Beam Correction Factor	59
4.5.6.4	Corrections for Angular Sensitivity	59
4.5.7	Average Glandular Dose, AGD (MGD)	60
4.5.8	Mammographic Pre-pulses	60
4.5.9	Scanning Beam Mammography	61
4.6	Dental and Panoramic Dental	61
4.6.1	Waveforms	63
4.7	CT	63
4.7.1	CT kVp	63
4.8	Tube Current Probes	65
4.8.1	MAS-1, Invasive mAs Probe	66
4.8.2	MAS-2, Non-invasive mAs Probe	67
4.9	Light Measurement	68
4.9.1	Luminance - Monitor/View box (cd/m ²)	69
4.9.2	Illuminance - Ambient Light (lx)	69
5.	Problems and Solutions.....	70
5.1	Troubleshooting	71
5.2	Bluetooth	72
5.2.1	Bluetooth Passkey	73
5.2.2	Enable Bluetooth Passkey	73
5.3	Windows Restricted User Accounts	74
5.3.1	Which Applications are Needed?	74
5.3.2	Installing the Softw are	75
5.3.3	Setting Necessary Rights	75
5.4	How To Report a Problem	76
6.	Glossary.....	77
	Index	91

Chapter 1

Introduction

1 Introduction

1.1 About this Manual

This manual is divided into a few main parts.

1. A general description of the Piranha.
2. Some theoretical background and basic principles.
- 3-4. Descriptions on performing measurements with the system for different modalities.
5. Description of different accessories for the Piranha.
6. Troubleshooting tips, an FAQ, and a glossary.

Note! This manual is written for Piranha with product version 5.7 (black top label), but can in general also be used for older Piranhas. Note however, that specifications may be more specific, and you should verify the specifications with those given in the manual delivered with your meter (corresponding to your Piranha product version on the bottom label).

Users who use the Piranha with only a PC and Ocean are recommended to read at least the following topics:

- Introduction
- Description of the Piranha
- Measurements with the Piranha System

For the Piranha system, calibration data is stored inside the system. See section [Managing Detector Calibrations](#) ^[34] for more information.

Typographical Rules

Terms in **bold face** are references to texts on screenshots, like buttons and texts, and menu items. Other terms are *italicized*.

1.2 Introduction to the Piranha

Congratulations to your purchase of a Piranha. You have now in your hand the most powerful tool for X-ray analysis. It has been carefully designed to meet the needs of both standard QA applications as well as advanced service/repair/calibration of modern X-ray systems, while still being very simple and intuitive to use. It can measure all the required parameters such as kVp, exposure time, dose, HVL, Total Filtration, dose/pulse, dose rate, tube current, mAs, waveforms, and much more. The Piranha can be used in two different ways:

- As a "meter" with a PC and the Ocean Quick-Check software.
- As a complete "QA-system" with a PC and the Ocean software.

This manual describes the Piranha. The PC software, Ocean, and the older QABrowser are described in detail in separate manuals.

The Piranha system's main features are:

- Very easy and intuitive to use
- Accurate
- Active Compensation - No manual corrections are needed
- Measures on all modalities with one detector
- Specially designed measuring modes for pulsed waveforms
- Compact
- Ocean is used for control and data processing
- Waveform analyser
- USB and Bluetooth interface
- Free upgrade of firmware
- Unique design

Free upgrades of the firmware (the software resident in the cabinet and measuring modules) are available on RTI Web site at <http://www.rtigroup.com>.

If you have questions, comments, or feel that some functionality is missing, you are welcome to contact us at RTI at support@rtigroup.com. You can of course also call (see notice section for details).

1.3 PC Requirements

To run the RTI Updater, the RTI Detector Manager, and Ocean the following is required:

Minimum requirements

Windows XP, 2003, Vista, 7/8/10 32-bit, or 7/8/10 64-bit.

Pentium class 300 MHz, 64 MB RAM (24 MB free), 60 MB of HD ¹

USB port

Display and graphics card with at least 800×600 resolution

Recommended requirements

Windows 7/8/10 32-bit or 8/10 64-bit

Pentium class 500 MHz, 128 MB RAM (32 MB free), 100 MB HD

USB port

CD/DVD-ROM for installation

Internet connection for updates (Recommended)

Note!

¹: Virtual memory and available hard drive space. Microsoft recommends that you have at least 20 % of your total HD space free for virtual memory.

Chapter 2


Description of the Piranha

2 Description of the Piranha

2.1 Indicators and Connectors


The Piranha comes in a lot of different models, the external design is basically the same for all models (except for the External Probe port).

Edge: External Probe port
(on some Piranha models)

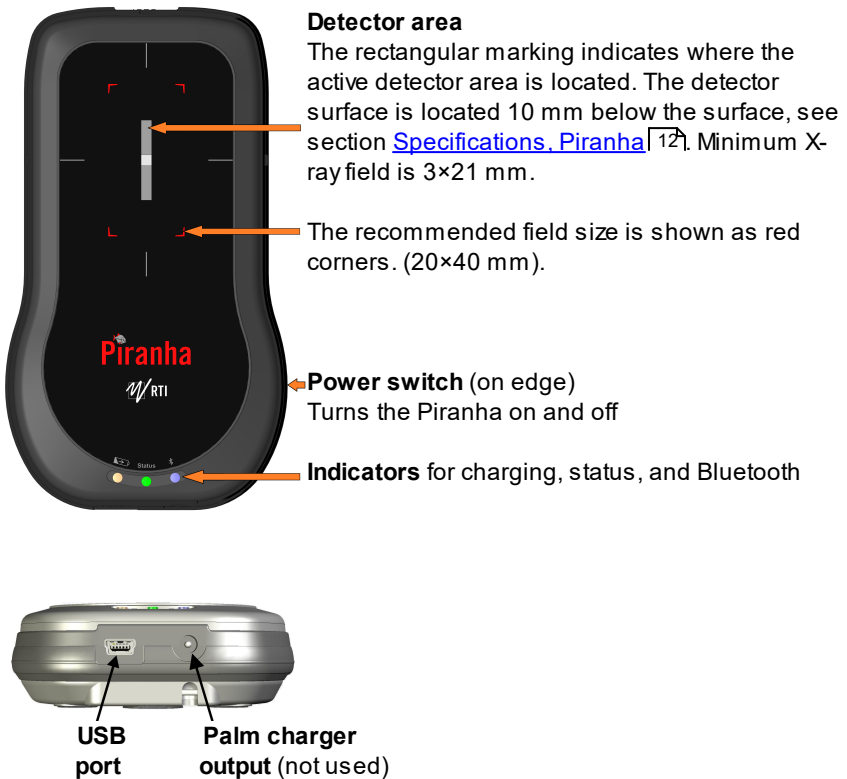

The **USB port** is used when using RTI Updater to update the internal firmware. It can also be used when the Piranha is used together with a PC running the QA software Ocean. Note that the USB connector cannot be used when connecting to a handheld computer. The system is then powered from the PC via the USB cable. The PC however have a limited USB power output, so when fast charging is needed the power supply needs to be connected here. This is also possible when using the Bluetooth link to communicate with the Palm or PC. The port is marked **USB**.

The orange indicator for **Charging of batteries** is lit when charging is active. Note that charging is possible even when the power switch is off.

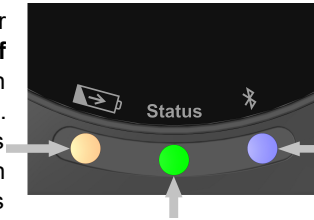

The blue indicator for **Bluetooth** is lit when the Bluetooth interface is active and discoverable.

The multi-coloured indicator for **Status** shows the status of the system, e.g. battery level as described below. Also works as **Power indicator**, one of the colours will light when the Piranha is on.

Battery level

The status indicator is used to show the battery level of the Piranha.

- Starting a system running on batteries the status indicates for 3 seconds:
 - Green if battery level over 25 % (4 h left)
 - Yellow if battery level between 10 and 25 % (1½-4 h left)
 - Red if battery level below 10 % (<1½ h left).

The idea is to get a quick indication when powering on the system if it will take me through today's work.

- When running on batteries the status indicator shows:

Status colour	Running time left
Green	>2 hours
Yellow	>1 hours
Red	>15 minutes
Flashing red	<15 minutes

Note! You may also check the battery level in the software. For Ocean on the PC, please see the Ocean manual.

See section [Power & Communication Specifications](#) ^[12] for more information about battery charging and discharging times.



External Probe port and Opening for filter position viewing

Here you attach the external probes that come with some models of the Piranha. The port is marked **EXT**. **Note!** Not all models have this port. The small square opening above may be used for monitoring the filter position inside the Piranha.



Power switch
Turns the Piranha
on and off (Marked
1/0)


Product marking

Indicates the model of your Piranha, as well as the version, serial number, and applicable conformity markings.

Attachment for **Safety strap**
Camera thread for mounting
the Piranha to a holder.

The **Power switch** is used to turn the Piranha on and off. Piranha has several ways of saving power when it is inactive, but must be powered off manually since there is no auto-power off function.

Below a block diagram of a typical Piranha system is shown.

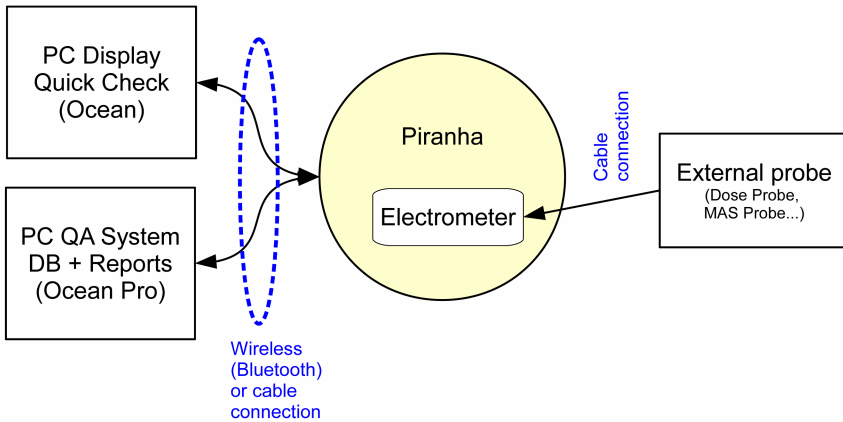

2.2 Setting Up the Piranha for the First Time

Before you use your Piranha for the first time, please do the following:

- Attach the external power supply.
- Charge the system for 16 hours.

Then continue according to the following section.

2.3 Setting Up the Piranha

To set up the Piranha:

1. Power on the Piranha using the power switch. Optionally you may connect the power supply.
2. Place the Piranha under the tube or mount the holder and HVL stand for positioning of the Piranha in the X-ray field. The stand allows you to position the Piranha (or the external Dose Probe) and HVL filters in any angle including upside-down. Use the light-field or other help to position the Piranha in the X-ray field. The Piranha detector is not sensitive for different field sizes as long as the entire sensitive detector area is irradiated, but try to keep the field size down to minimize scattering.

It is recommended to position the Piranha in such a way that the detector area is orientated perpendicular to the anode/cathode axis, to avoid the heel effect.

Note!

Adjust the collimator so the radiation clearly covers the detector rectangle marked on the Piranha top panel, but try to keep the field size inside the top panel size to minimize scatter. Recommended field size is 20×40 mm. Furthermore the Piranha top surface should

optimally be placed perpendicular to the focal spot, see also [Angular Sensitivity, Piranha](#) ²³⁾

3. Connect with Ocean via included USB cable or via Bluetooth.

2.4 Hardware and Specifications


Specifications are valid after a warm-up time of one minute and presuming reference conditions. All specifications are for use together with the Piranha unless otherwise stated. All specifications can be changed without prior notice. RTI Group AB assumes no responsibility for any errors or consequential damages that may result from the misuse or misinterpretation of any information contained in these specifications.

2.4.1 Piranha internal detector (Internal detector)

2.4.1.1 General

With the Piranha internal detector you will manage most of your measurements. Tube voltage, exposure time, dose, and dose rate are measured for all kinds of modalities: conventional radiography, fluoroscopy, pulsed fluoroscopy, cine, mammography, dental, panoramic dental, and CT (kVp only, not dose and dose rate). In one exposure, the detector provides tube voltage, time, dose, dose rate, quick-HVL, and estimated total filtration on radiographic, fluoroscopic, dental, and CT exposures. On pulsed radiation and cine, also dose per pulse and pulse rate are measured. The Piranha internal detector is very sensitive and can measure peak tube voltage for as low outputs as 50 kV / 0.050 mA at 50 cm.

Typically the exposure time has to be at least 5 ms to get a kVp value but it depends on the waveform. On modern X-ray generators (high-frequency with fast rise and fall times) the peak tube voltage can normally be measured with exposure time as short as 1 ms. Dose and time values will be given for even shorter exposure times. The estimations of total filtration and Quick-HVL are done from one single exposure using a combination of detector and filters in the Piranha. In situations when the total filtration cannot be automatically estimated, a "standard" HVL measurement may be required. All measured kVp and dose values measured with the Piranha are automatically compensated for the actual beam/radiation quality. This means that no manual corrections of measured data is needed.

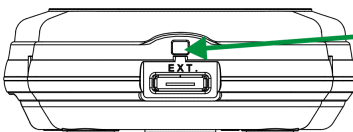

The range indicator can be viewed behind a little lid, that can be pushed to the side. Make sure to close it afterwards, to avoid light leaking into the detector.

2.4.1.2 Power & Communication Specifications

Power Source

Power supply	5 V AC/DC adapter w ith Mini-USB connector, internal battery, or USB cable supply from PC.
Battery operated	One 2000 mAh Li-Ion battery. Operation time typically 15 hours. Typical charging times are listed below .
External power	100-240 V AC 50/60 Hz w ith external adapter.
Power output	On connector marked "5V OUT" for supply/charging of Palm. Only functional w hen AC/DC adapter is connected to USB port.

Typical Battery Charging and Running Times

Capacity	Running time	Charging time		
		Using Power supply	USB, Piranha ON	USB, Piranha OFF
50 %	7½ h	1½ h (90 min)	3½ h	17 h
80 %	12 h	2½ h (150 min)	6 h	27 h
90 %	13½ h	3¼ h (195 min)	7 h	30 h
100 %	15 h	5 h (300 min)	8½ h	32 h

Note that other mains power solutions that uses a regular USB cable to connect to the Piranha, will behave like USB in the table.

Communication

USB	Max 12 Mbit/s (USB v1.1)
Bluetooth	115 kbit/s

2.4.1.3 Specifications, Piranha

The inaccuracy is here defined as the root of the square sum of systematic errors, which has not been eliminated, and random errors (dispersion around a mean value). The calculation of the inaccuracy is based on 15 different measurements and with a confidence level of 95 %. Of the total inaccuracy, random error is 20 % and general inaccuracy is 80 %.

Note: Irradiation time is often called exposure time in daily use.

General

Operating temperature and relative humidity	15 – 35 °C at <80 % relative humidity
Storage temperature	-10 °C to +50 °C
Operating air pressure	Minimum 80 – 106 kPa
Source to Detector Distance	>30 cm


Reference conditions

Temperature	20 °C
Relative humidity	50 %
Air pressure	101.3 kPa
X-ray field size	Inside the Piranha top panel. Calibration is done with field size typically 5 mm less than the size of the top panel.
Radiation quality	
Radiography	70 kV, 2.5 mm Al
Mammography	28 kV, 30 µm Mo
CT	120 kV, 2.5 mm Al

Note: The reference conditions are given in reference to the IEC61674 standard.

Physical dimensions

Detector area	3 × 21.1 mm
Detector position	10 mm below top panel, as indicated in figure below and by a 3 mm rim on 3 edges.
Size	133 × 75 × 26 mm (5.2" × 2.9" × 1.02")
Weight	Approximately 405 g

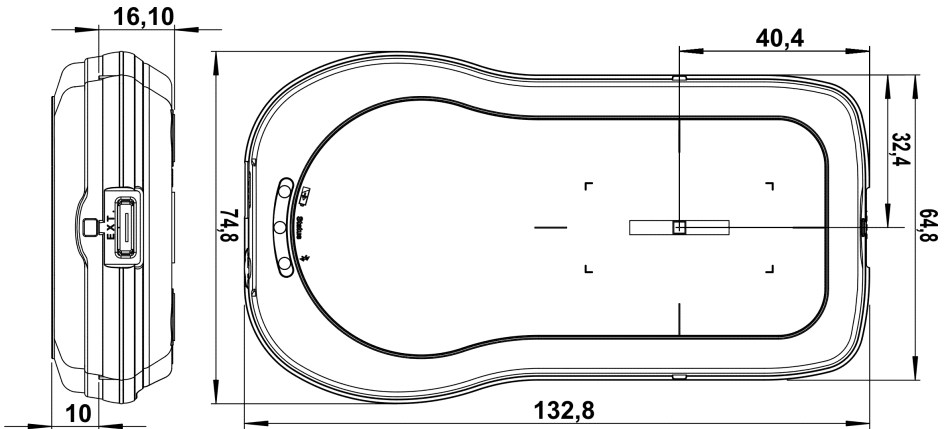

Parameters

Tube voltage (kVp)	The average of all samples with compensation for the ripple (default method)
Time	Irradiation time (Exposure time)
Air kerma (Dose)	Measured air kerma (may be called dose or air kerma in this manual)
Air kerma rate (Dose rate)	Average air kerma rate (may be called dose rate or air kerma rate in this manual)
Total Filtration	Estimation of total filtration (for conventional radiography, fluoroscopy, dental, and CT)
Quick-HVL	Estimation of Half Value Layer (for conventional radiography, fluoroscopy, dental, mammography, and CT)
Half Value Layer	Standard HVL using filters for evaluation on radiography, fluoroscopy, dental, and mammography (all for both pulsed and conventional)
kV waveform	Waveform is calculated based on detector signals measured after different thickness of filtration.
Dose rate waveform	Signal measured from radiation detector (ionization chamber or solid-state detector).

Measuring range and inaccuracy

These specifications are written for Piranha with product version 5.7 (black top label), but can in most part also be used for older Piranhas. Note however that for older versions, you should verify the specifications with those given in the manual delivered with your meter (corresponding to your Piranha product version on the bottom label).

Note!

Radiography, Fluoroscopy, and Dental

Parameter	RQ	Range	Inaccuracy ²	Resolution ²
kVp (standard) W / 3 mm Al	R1	35 – 160 kV	±1.5 %	4 digits (10 or 100 V)
kVp dental W / 3 mm Al	R1	35 – 125 kV	±1.5 %	As above
Irradiation time		0.1 ms – 2000 s 1 – 65535 pulses	±0.5 % or ±0.5 ms ±1 pulse	0.5 ms 1 pulse
Air kerma (Dose) ²		1.3 nGy – 650 Gy (150 nR – 74 kR)	±5 %	–
Air kerma rate ^{2 3} (Dose rate)		15 nGy/s – 320 mGy/s 1.7 µR/s – 37 R/s 0.1 mR/min – 2200 R/min	±5 % or ±7 nGy/s ±5 % or ±0.8 µR/s ±5 % or ±0.05 mR/min (for Irr. time >20 ms)	Typ. noise: ±3 nGy/s WF@2kSa/s : ±4 µGy/s
Per Measuring mode: -Free run / Timed		15 nGy/s – 11 mGy/s	±5 % or ±7 nGy/s	Typ. noise:

Parameter	RQ	Range	Inaccuracy ²	Resolution ²
-Normal (High Sens.)		330 nGy/s – 11 mGy/s	±5 % or ±7 nGy/s	±3 nGy/s
-Normal (Low Sens.)		25 µGy/s – 320 mGy/s	±5 % or ±0.1 µGy/s	
Estimated total filtration		1.0 – 90 mm Al (full kV range)	±10 % or ±0.3 mm ±15 % >50 mm Al (60 – 120 kV, HF/DC, >10 µGy/s)	2 digits (0.1 or 1 mm)
Quick-HVL		1.2 – 14 mm Al ⁴ (35 – 150 kV, TF=1-45 mmAl)	±10 % or ±0.2 mm (60 – 120 kV, HF/DC, >10 µGy/s) ¹	3 digits (0.01 or 0.1 mm)

Note 1: This is valid for a tube with 14° anode angle. The HVL for a 22° anode is typically 0,5 mm lower (@ 80 kV, 3 mm TF).

Note 2: All kerma and kerma rate ranges, inaccuracy, and resolution figures are valid for product version 5 and higher of the Piranha.

Note 3: The Kerma rate is calculated as the Kerma (Dose) divided by the Irradiation time. See also [Waveforms and Triggers](#) ^[47].

Note 4: The HVL range is valid if also the TF is within its specified range. For high TF at high kV the HVL range may be limited by this.

Mammography

Parameter	RQ	Range	Inaccuracy	Resolution
kVp (standard)				
Mo / 30 µm Mo	M1	18 – 49 kV	±1.5 % or ±0.7 kV	4 digits (10 V)
Mo / 25 µm Rh	M3	20 – 46 kV	±2 % or ±1 kV	
Rh / 25 µm Rh	M4	25 – 49 kV	±2 % or ±1 kV	
Rh / 1.0 mm Al	M5	22 – 35 kV	±2 % or ±1 kV	
W / 50 µm Rh ³	M6 ³	20 – 49 kV	±2 % or ±1 kV	
W / 0.50 mm Al ⁵	M7 ⁵	20 – 48 kV	±2 % or ±1 kV	
Mo / 1.0 mm Al	M8	18 – 49 kV	±2 % or ±1 kV	
W / 50 µm Ag ³	M10 ³	20 – 40 kV	±2 % or ±1 kV	
W / 75 µm Ag	M11	20 – 40 kV	±2 % or ±1 kV	
W / 50 µm Rh (Gio)	M12	22 – 35 kV	±2 % or ±1 kV	
W / 0.70 mm Al ³	M15 ³	20 – 49 kV	±1.5 % or ±0.7 kV	
W / 50 µm Ag (Sel) ⁴	M16 ⁴	22 – 39 kV	±2 % or ±1 kV	
W / 50 µm Rh (Sel) ⁴	M17 ⁴	22 – 39 kV	±2 % or ±1 kV	
W / 0.30 mm Cu ³	M18 ³	40 – 49 kV	±2 % or ±1 kV	
W / 0.70 mm Al (Inno/Crist) ⁶	M19 ⁶	20 – 49 kV	±2 % or ±1 kV	
W / 50 µm Rh (Inno/Crist) ⁶	M20 ⁶	20 – 49 kV	±2 % or ±1 kV	
Mo / 25 µm Rh (Sel) ⁴	M21 ⁴	20 – 46 kV	±2 % or ±1 kV	
Rh / 30 µm Ag (GE) ⁷	M22 ⁷	27 – 40 kV	±2 % or ±1 kV	
Rh / Ag IQST (GE) ⁷	M23 ⁷	31 – 37 kV	±2 % or ±1 kV	
Mo / 0.25 mm Cu (GE) ⁷	M24 ⁷	40 – 49 kV	±2 % or ±1 kV	
Rh / 0.25 mm Cu (GE) ⁷	M25 ⁷	40 – 49 kV	±2 % or ±1 kV	
Mo / 30 µm Mo (GE) ⁷	M26 ⁷	22 – 32 kV	±2 % or ±1 kV	
Affirm Prone W / 50 µm Ag ⁸	M27 ⁸	20 – 40 kV	±2 % or ±1 kV	
Affirm Prone W / 0.70 mm Al ⁸	M28 ⁸	20 – 49 kV	±2 % or ±1 kV	

Note 1: The calibrations above M12 are available only for product version 5 and higher of the Piranha.

Note 2: Notes 3-9 below, are valid only for product version 5 and higher of the Piranha.

Note 3: The M6 (*W / 50 μ m Rh*), M10 (*W / 50 μ m Ag*), M15 (*W / 0.70 mm Al*), and M18 (*W / 0.30 mm Cu*) calibrations are suitable for the Hologic Selenia Dimensions and Fuji Amulet (M6, M15 only).

Note 4: The M16 (*W / 50 μ m Ag (SeI)*), M17 (*W / 50 μ m Rh (SeI)*), and (M21 *Mo / 25 μ m Rh (SeI)*) calibrations are suitable for Hologic Selenia.

Note 5: The M7 (*W / 0.5 mmAl*) calibration is suitable for Philips MicroDose Mammography (Sectra).

Note 6: The M19 (*W / 0.70 mm Al*) and M20 (*W / 50 μ m Rh*) calibrations are suitable for Fujifilm FDR MS-2000/2500/3000 (Innovality/Cristalle).

Note 7: The M22 to M26 (*Mo / Rh combinations*) calibrations are suitable for the GE Senographe Pristina™.

Note 8: The M27 (*W / 50 μ m Ag*) and M28 (*W / 0.70 mm Al*) are optimized for and dedicated to Hologic Affirm Prone.

Note 9: The M18 to M28 calibrations are visible also when using the QABrowser, but cannot be used.

Parameter	Range	Inaccuracy ¹	Resolution ¹
Irradiation time	0.1 ms – 2000 s 1 – 65535 pulses	±0.5 % or ±0.5 ms ±1 pulse	0.5 ms 1 pulse
Air kerma (Dose) ¹	1 nGy – 1000 Gy 3 μ R – 150 kR	±5 % ±5 %	–
Air kerma rate ^{1 2} (Dose rate)	25 nGy/s – 530 mGy/s 30 μ R/s – 60 R/s 1.8 mR/min – 3600 R/min	±5 % or ±12 nGy/s ±5 % or ±1.5 μ R/s ±5 % or ±0.1 mR/min (for Irr. time >20 ms)	Typ. noise: 6 nGy/s WF@2kSa/s: 8 μ Gy/s
Per Measuring mode:			
-Free run / Timed	25 nGy/s – 20 mGy/s	±5 % or ±12 nGy/s	Typ. noise:
-Normal (High Sens.)	0.54 μ Gy/s – 20 mGy/s	±5 % or ±12 nGy/s	6 nGy/s
-Normal (Low Sens.)	40 μ Gy/s – 530 mGy/s	±5 % or ±0.2 μ Gy/s	

Note 1: All kerma and kerma rate ranges, inaccuracy, and resolution figures are valid for product version 5 and higher of the Piranha.

Note 2: The Kerma rate is calculated as the Kerma (Dose) divided by the Irradiation time. See also [Waveforms and Triggers](#) ⁴⁷.

Parameter	RQ	Range	Inaccuracy	Resolution
Quick-HVL ^{7 8}				
Mo / 30 μ m Mo	M1	0.19 – 0.47 mm Al	±10 %	3 digits (0.001 mm)
Mo / 25 μ m Rh	M3	0.31 – 0.52 mm Al		
Rh / 25 μ m Rh	M4	0.33 – 0.60 mm Al		
Rh / 1.0 mm Al	M5	0.31 – 0.68 mm Al		
W / 50 μ m Rh ¹	M6 ¹	0.37 – 0.75 mm Al		
W / 0.50 mm Al ³	M7 ³	0.24 – 0.64 mm Al		
Mo / 1.0 mm Al	M8	0.31 – 0.68 mm Al		
W / 50 μ m Ag ¹	M10 ¹	0.34 – 0.69 mm Al		
W / 75 μ m Ag	M11	0.41 – 0.79 mm Al		
W / 50 μ m Rh (Gio)	M12	0.37 – 0.66 mm Al		
W / 0.70 mm Al ¹	M15 ¹	0.27 – 0.81 mm Al		

Parameter	RQ	Range	Inaccuracy	Resolution
W / 50 μ m Ag (Sel) ²	M16 ²	0.34 – 0.69 mm Al		
W / 50 μ m Rh (Sel) ²	M17 ²	0.37 – 0.66 mm Al		
W / 0.30 mm Cu ¹	M18 ¹	2.43 – 4.00 mm Al		
W / 0.70 mm Al (Inno/Crist) ⁴	M19 ⁴	0.27 – 0.81 mm Al		
W / 50 μ m Rh (Inno/Crist) ⁴	M20 ⁴	0.37 – 0.75 mm Al		
Mo / 25 μ m Rh (Sel) ²	M21 ²	0.31 – 0.52 mm Al		
Rh / 30 μ m Ag (GE) ⁵	M22 ⁵	0.32 – 0.71 mm Al		
Rh / Ag IQST (GE) ⁵	M23 ⁵	0.9 – 1.3 mm Al		
Mo / 0.25 mm Cu (GE) ⁵	M24 ⁵	3.7 – 4.3 mm Al		
Rh / 0.25 mm Cu (GE) ⁵	M25 ⁵	3.7 – 4.3 mm Al		
Mo / 30 μ m Mo (GE) ⁵	M26 ⁵	0.19 – 0.47 mm Al		
Affirm Prone W / 50 μ m Ag ⁶	M27 ⁶	0.34 – 0.69 mm Al		
Affirm Prone W / 0.70 mm Al ⁶	M28 ⁶	0.27 – 0.81 mm Al		

Note 1: The M6 (W / 50 μ m Rh), M10 (W / 50 μ m Ag), M15 (W / 0.70 mm Al), and M18 (W / 0.30 mm Cu) calibrations are suitable for the Hologic Selenia Dimensions and Fuji Amulet (M6, M15 only).

Note 2: The M16 (W / 50 μ m Ag (Sel)), M17 (W / 50 μ m Rh (Sel)), and (M21 Mo / 25 μ m Rh (Sel)) calibrations are suitable for Hologic Selenia.

Note 3: The M7 (W / 0.5 mm Al) calibration is suitable for Philips MicroDose Mammography (Sectra).

Note 4: The M19 (W / 0.70 mm Al) and M20 (W / 50 μ m Rh) calibrations are suitable for Fujifilm FDR MS-2000/2500/3000 (Innovality/Cristalle).

Note 5: The M22 to M26 (Mo / Rh combinations) calibrations are suitable for the GE Senographe Pristina™.

Note 6: The M27 (W / 50 μ m Ag) and M28 (W / 0.70 mm Al) are optimized for and dedicated to Hologic Affirm Prone.

Note 7: The Quick-HVL for mammography is only available for Piranhas with product version 2 and higher.

Note 8: The M18 to M28 calibrations are visible also when using the QABrowser, but cannot be used.

Computed Tomography

Parameter	RQ	Range	Inaccuracy	Resolution
kVp (standard)				
W / 3.0 mm Al	C1	45 – 155 kV	±1.5 %	4 digits (10 or 100 V)
W / 3 mm Al + 1.2 mm Ti (Siemens Straton (Siem1)) ²	C3 ₂	75 – 145 kV		
GECT (7°) ^{2,3}	C4 ₂	75 – 145 kV		
Acquillion 64 (Toshiba) ²	C5 ₂	75 – 145 kV		
GECT (10.5°) ^{2,3}	C6 ₂	65 – 150 kV		
Irradiation time		0.1 ms – 2000 s 1 – 65535 pulses	±0.5 % or ±0.5 ms ±1 pulse	0.5 ms 1 pulse
Air kerma (Dose)		4	4	4
Estimated total filtration	C1	1.0 – 90 mm Al (full kV range)	±10 % or ±0.3 mm ±15 % >50 mm Al (75 – 120 kV, HF/DC, >10 µGy/s)	2 digits (0.1 or 1 mm)
	C6	1.0 – 90 mm Al	±10 % or ±0,3 mm (65 – 150 kV, 2 – 10 mm Al, HF/DC, >10 µGy/s)	
Quick-HVL	C1	1.2 – 14 mm Al (45 – 150 kV)	±10 % or ±0.2 mm (75 – 120 kV, HF/DC, >10 µGy/s) ¹	3 digits (0.01 or 0.1 mm)

Note 1: This is valid for a tube with 14° anode angle. The HVL for a 22° anode is typically 0,5 mm lower (@ 80 kV, 3 mm TF).

Note 2: The C3 and higher numbered calibrations are only available for product versions 2.0 or higher.

Note 3: The C4, C5, and C6 are suitable for GE CT tubes. C4 (GECT (7°)) is suitable for all GE CT tubes which have a 7° anode angle as well as other manufacturers CT tubes and replacement tubes with a 7° anode angle. The C6 (GECT (10.5°)) is suitable for GE CT tubes with a 10.5° anode angle.

Note 4: CT dose is measured with a CT ionization chamber connected to a Piranha Chamber Adapter. See Piranha Chamber Adapter manual for details.

Pulses

Parameter	Range
Dose/pulse	8 nGy/pulse - 0.9 Gy/pulse ¹
Pulse dose rate	Lower limit 10 μ Gy/s (70 mR/min), otherwise same as for air kerma rate.
Min. output peak dose rate - High Sensitivity - Low Sensitivity	dose rate (min. pulse width) 4 μ Gy/s (4 ms) / 30 μ Gy/s (0.5 ms) 20 μ Gy/s (4 ms) / 160 μ Gy/s (0.5 ms)
Pulse rate	0.5 – 180 Hz, resolution 0.5 Hz
Pulse width	4 ms - 2000 s
Duty cycle	5 - 95 %
Minimum pulse width - High Sensitivity - Low Sensitivity	pulse width (min. dose rate) 4 ms (4 μ Gy/s) / 0.5 ms (30 μ Gy/s) 4 ms (20 μ Gy/s) / 0.5 ms (160 μ Gy/s)
Minimum ripple (pulse top to bottom)	50 %
Irradiation time	1 – 65535 pulses, resolution 1 pulse

Note 1: Max dose/pulse depends on the pulse length.

Note 2: All kerma and kerma rate ranges, inaccuracy, and resolution figures are valid for product version 2 and higher of the Piranha.

Waveform recording time

At max sampling rate	1024 ms (2 kSa/s)
At min sampling rate	524 s (4 Sa/s)

A total of 8 recording times are available, all separated by a factor of 2, i.e. 1, 2, 4, 8, 16, 33, 66, 131, 262 and 524 seconds.

Note!

The setting for *Waveform recording time* may affect the Irradiation time calculation. Make sure to set back the *Waveform recording time* to the lowest choice after temporarily modifying it.

2.4.1.4 Typical Response, Piranha

The table below shows the typical response for the Piranha at standardised radiation qualities.

Radiography, Fluoroscopy, and Dental (measured using RTI RQ Code R1, W/Al)

Radiation quality		Mean energy air kerma (keV)	Total Filtration (mm Al)	Air kerma measurement	
PTB	ISO 4037 IEC 61267			HVL (mm Al)	Factor kQ (Rel. RQR 5)
DV40	RQR 2	26,3	2,49	1,42	0,9959
DV50	RQR 3	28,8	2,46	1,77	1,0073
DV60	RQR 4	31,7	2,68	2,19	1,0033
DV70	RQR 5	34,9	2,83	2,57	1
DV80	RQR 6	37,5	2,99	3,01	0,9956
DV90	RQR 7	40,7	3,18	3,48	0,9931
DV100	RQR 8	43,8	3,36	3,96	0,9957
DV120	RQR 9	50,3	3,73	5,00	1,0008
DV150	RQR 10	60,7	4,38	6,55	1,0132

Note: These values are typical values measured at PTB in Germany in 2014.

Radiation quality		Mean energy air kerma (keV)	Total Filtration (mm Al)	Air kerma measurement	
PTB	ISO 4037 IEC 61267			HVL (mm Al)	Factor kQ (Rel. RQR 5)
DH40	RQA 2	30,22	6,49	2,2	1,0321
DH50	RQA 3	37,76	12,5	3,74	1,0043
DH60	RQA 4	44,74	18,7	5,32	0,9953
DH70	RQA 5	51,08	23,8	6,73	1
DH80	RQA 6	57,47	29,0	8,12	1,0041
DH90	RQA 7	62,95	33,2	9,21	1,0214
DH100	RQA 8	68,19	37,4	10,10	1,0147
DH120	RQA 9	78,36	43,7	11,59	1,0129
DH150	RQA 10	93,36	49,4	13,23	1,0377

Note: These values are typical values measured at PTB in Germany in 2014.

Mammography, Mo / 30 µm Mo and 30 µm Mo + 2 mm Al (measured using RTI RQ Code M1)

Radiation quality		Mean energy air kerma (keV)	Air kerma measurement	
PTB	ISO 4037 IEC 61267		HVL (mm Al)	Factor kQ (Rel. RQR-M2)

MMV25	RQR-M1	14,86	0,28	0,9869
MMV28	RQR-M2	15,39	0,31	1
MMV30	RQR-M3	15,67	0,33	1,0068
MMV35	RQR-M4	16,24	0,37	1,0067
MMH25	RQA-M1	18,28	0,55	0,9945
MMH28	RQA-M2	18,88	0,60	1,0091
MMH30	RQA-M3	19,34	0,63	1,0058
MMH35	RQA-M4	20,55	0,71	0,9885

Note: These values are typical values measured at PTB in Germany in 2014.

Mammography, Mo / 1 mm Al (measured using RTI RQ Code M8)

Radiation quality		Mean energy air kerma (keV)	Air kerma measurement	
PTB	ISO 4037 IEC 61267		HVL (mm Al)	Factor kQ (Rel. MAV28)
MAV25	-	17,58	0,48	0,989
MAV28	-	18,29	0,54	1
MAV30	-	18,66	0,56	0,9978
MAV35	-	19,36	0,61	0,9944
MAV40	-	19,89	0,64	1,006

Note: These values are typical values measured at PTB in Germany in 2009 and 2014.

Mammography, Mo / 25 µm Rh (measured using RTI RQ Code M3)

Radiation quality		Mean energy air kerma (keV)	Air kerma measurement	
PTB	ISO 4037 IEC 61267		HVL (mm Al)	Factor kQ (Rel. MRV28)
MRV25	-	15,78	0,34	0,996
MRV28	-	16,29	0,38	1
MRV30	-	16,54	0,39	0,9980
MRV35	-	17,02	0,43	0,9911
MRV40	-	17,4	0,45	1,001

Note: These values are typical values measured at PTB in Germany in 2009 and 2014.

Mammography, Rh / 25 µm Rh (measured using RTI RQ Code M4)

Radiation quality		Mean energy air kerma (keV)	Air kerma measurement	
PTB	ISO 4037 IEC 61267		HVL (mm Al)	Factor kQ (Rel. RRV28)
RRV25	-	15,57	0,32	0,996
RRV28	-	16,34	0,37	1
RRV30	-	16,73	0,39	1,0036
RRV35	-	17,57	0,45	1,0089
RRV40	-	18,18	0,49	1,005

Note: These values are typical values measured at PTB in Germany in 2009 and 2014.

Mammography, W / 0.5 mm Al (measured using RTI RQ Code M7)

Radiation quality		Mean energy air kerma (keV)	Air kerma measurement	
PTB	ISO 4037 IEC 61267		HVL (mm Al)	Factor kQ (Rel. WAV28)
WAV25	-	16,08	0,35	0,979
WAV28	-	16,97	0,40	1
WAV30	-	17,49	0,43	0,9974
WAV35	-	18,73	0,51	0,9928
WAV40	-	19,79	0,58	0,982

Note: These values are typical values measured at PTB in Germany in 2009 and 2014.

Mammography, W / 50 µm Rh (measured using RTI RQ Code M6)

Radiation quality		Mean energy air kerma (keV)	Air kerma measurement	
PTB	ISO 4037 IEC 61267		HVL (mm Al)	Factor kQ (Rel. WRV28)
WRV25	-	17,6	0,48	0,997
WRV28	-	17,99	0,51	1
WRV30	-	18,19	0,52	1,0009
WRV35	-	18,78	0,56	0,9969
WRV40	-	19,54	0,61	1,022

Note: These values are typical values measured at PTB in Germany in 2009 and 2014.


Mammography, W / 50 μm Ag (measured using RTI RQ Code M10)

Radiation quality		Mean energy air kerma (keV)	Air kerma measurement	
PTB	ISO 4037 IEC 61267		HVL (mm Al)	Factor kQ (Rel. WSV28)
WSV25	-	17,87	0,50	0,991
WSV28	-	18,66	0,56	1
WSV30	-	18,92	0,58	0,9983
WSV35	-	19,57	0,63	0,9963
WSV40	-	20,22	0,68	1,015

Note: These values are typical values measured at PTB in Germany in 2009 and 2014.


2.4.1.5 Angular Sensitivity, Piranha

In this section you can see graphs of the typical angular sensitivity for dose measured with the Piranha at 28 and 70 kV. The setup is shown in figures below.

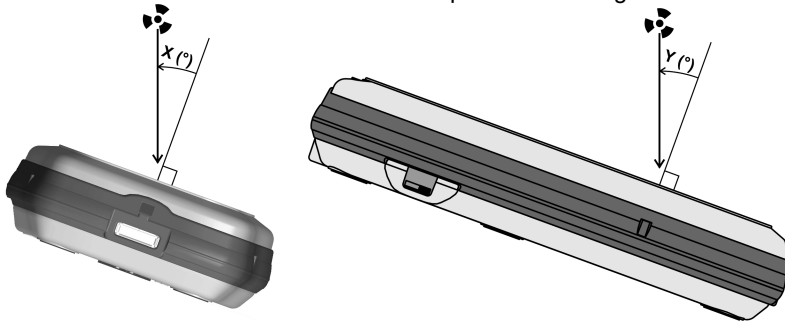


This "directional" behaviour makes it excellent for reproducible measurements, with less influence by nearby spreading matter. This makes it possible to make accurate HVL measurements even when measuring with "bad geometry", which is especially interesting for mammography. To understand, please see the polar plot shown below. The Piranha is shown to the left, and a typical mammographic ion chamber to the right.

Polar Sensitivity - 28 kV

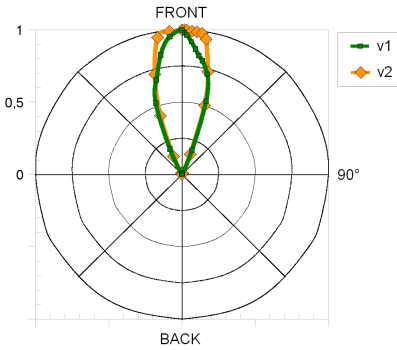

Polar Sensitivity - 28 kV Chamber


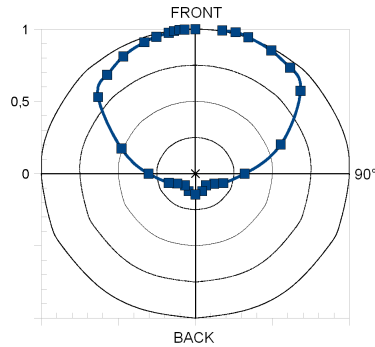



There are two different graphs, depending on the product version of your Piranha. The product version is the version number you can find on the label on the bottom of the Piranha. If the version of your Piranha is 1.X, use the graphs marked v1. For 2.X and higher use graphs marked v2.

Note!

For v1.X it is however important that you place the detector surface perpendicular to the direction of the radiation source or that you make corrections according to the tables in section [Corrections for Angular Sensitivity](#) ⁵⁹.

For radiography this is generally no problem, since most measurements are performed in the middle of the field, perpendicular to the incident radiation.

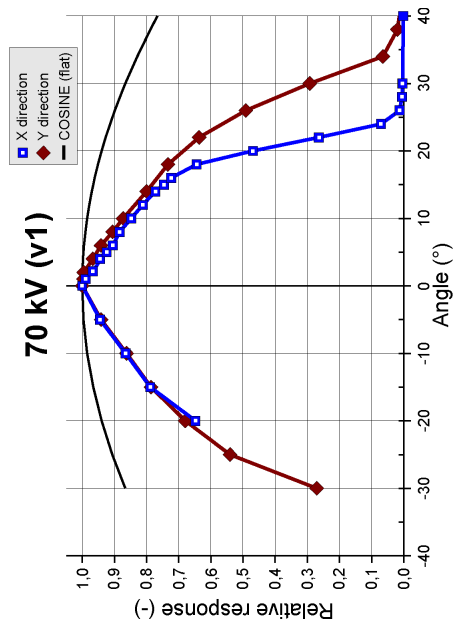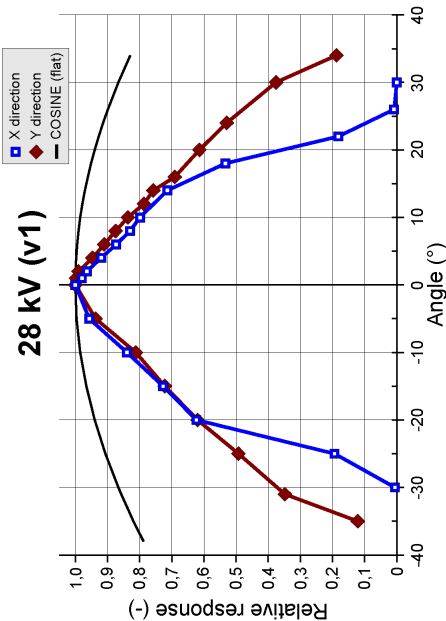

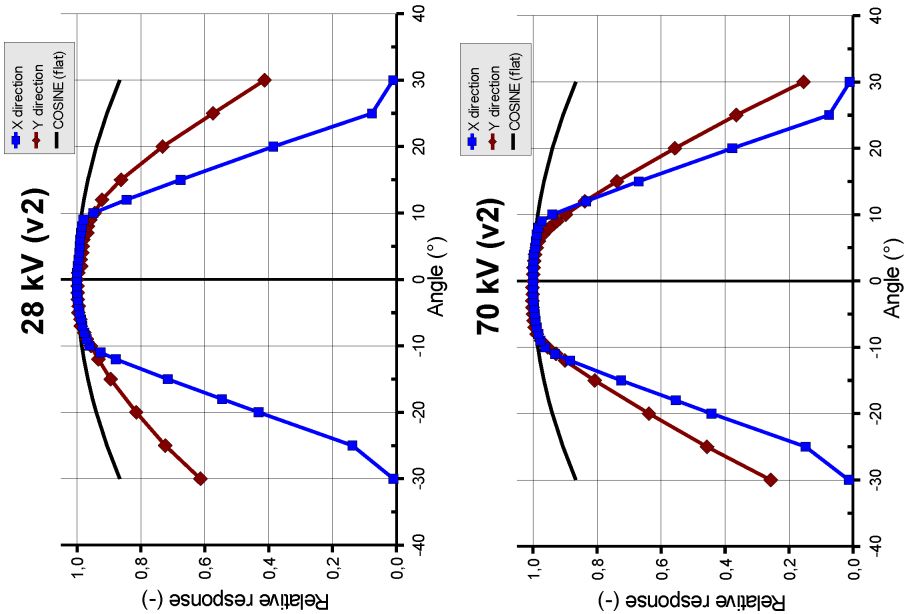

2.4.2 Piranha External Probes

The inaccuracy is here defined as the root of the square sum of systematic errors, which has not been eliminated, and random errors (dispersion around a mean value). The calculation of the inaccuracy is based on 15 different measurements and with a confidence level of 95 %. Of the total inaccuracy, random error is 20 % and general inaccuracy is 80 %.

Note: Irradiation time is often called exposure time in daily use.

Reference conditions

Temperature	20 °C
Relative humidity	50 %
Air pressure	101.3 kPa
Radiation quality	
Radiography	70 kV, 2.5 mm Al
Mammography	28 kV, 30 µm Mo
CT	120 kV, 2.5 mm Al

Note: The reference conditions are given in reference to the IEC61674 standard.

General

Connector type	Hirose ST40X-10S with built-in detector identification.
Source to Detector Distance	>30 cm

Measuring range and inaccuracy

The detector noise given is typical values at room temperature.

- Piranha/RTI External Dose Probe (typical sensitivity +55 $\mu\text{C}/\text{Gy}$)

Parameter	Range	Inaccuracy	Resolution
Air kerma (Dose)	100 pGy – 1.5 kGy 12 nR – 170 kR	$\pm 5\%$ (for time > 0.1 ms)	
Air kerma rate (Dose rate)	40 nGy/s – 150 mGy/s 4.6 $\mu\text{R}/\text{s}$ – 16 R/s 260 $\mu\text{R}/\text{min}$ – 1000 R/min 16 mR/h – 60 kR/h	(valid for Irr. time >20 ms) $\pm 5\%$ or ± 10 nGy/s $\pm 5\%$ or ± 1 $\mu\text{R}/\text{s}$ $\pm 5\%$ or ± 6 $\mu\text{R}/\text{min}$ $\pm 5\%$ or ± 360 $\mu\text{R}/\text{h}$	Typ. noise: ± 500 pGy/s WF@2kSa/s : ± 300 nGy/s
(Free-run or Timed) (+5 s moving average)	4 nGy/s – 150 mGy/s 1 nGy/s – 150 mGy/s	$\pm 5\%$ or ± 1 nGy/s $\pm 5\%$ or ± 250 pGy/s	± 200 pGy/s
Irradiation time	0.1 ms – 34000 s 1 – 65535 pulses	$\pm 1\%$ or ± 0.5 ms ± 1 pulse	0.5 ms

Note 1: The air kerma rate is calculated as the air kerma divided by the time. See also [Waveforms and Triggers](#) ^[47].

Note 2: The standard calibration for the Piranha External Dose Probe is W/23 mm Al. This calibration was chosen since the main use of the detector is to measure the dose to the image intensifier, after the phantom. However, you can just as well use this probe for measurements of skin dose. The detector is very linear in its energy response and will not be affected by a different filtration.

Pulses

Parameter	Range
Dose/pulse	1 nGy/pulse - 0.3 Gy/pulse ₁
Pulse dose rate	Lower limit 10 $\mu\text{Gy}/\text{s}$ (70 mR/min), otherwise same as for air kerma rate.
Min. output peak dose rate - High Sensitivity - Low Sensitivity	Doserate (min. pulse width) 0.23 $\mu\text{Gy}/\text{s}$ (4 ms) / 1.8 $\mu\text{Gy}/\text{s}$ (0.5 ms) 10 $\mu\text{Gy}/\text{s}$ (4 ms) / 73 $\mu\text{Gy}/\text{s}$ (0.5 ms)
Pulse rate - Normally	0.5 – 100 Hz, resolution 0.5 Hz
Pulse width	4 ms - 2000 s
Duty cycle	5 - 95 %
Minimum pulse width	pulse width (min. peak dose rate) 4 ms (0.23 $\mu\text{Gy}/\text{s}$) / 0.5 ms (1.8 $\mu\text{Gy}/\text{s}$)
Minimum ripple (pulse top to bottom)	50 %

Parameter	Range
Irradiation time	1 – 65535 pulses, resolution 1 pulse

Note 1: Max dose/pulse depends on the pulse length.

Waveform recording time

At max sampling rate	1024 ms (2 kSa/s)
At min sampling rate	524 s (4 Sa/s)

A total of 8 recording times are available, all separated by a factor of 2, i.e. 1, 2, 4, 8, 16, 33, 66, 131, 262 and 524 seconds.

Note! The setting for *Waveform recording time* may affect the Irradiation time calculation. Make sure to set back the *Waveform recording time* to the lowest choice after temporarily modifying it.

The table below shows the typical response for the Piranha/RTI External Dose Probe at standardised radiation qualities.

Radiation quality		Mean energy air kerma (keV)	Total Filtration (mm Al)	Air kerma measurement	
PTB	ISO 4037 IEC 61267			HVL (mm Al)	Factor kQ (Rel. RQR 5)
DV40	RQR 2	26,38	2,49	1,42	1,087
DV50	RQR 3	29,0	2,46	1,77	1,044
DV60	RQR 4	32,0	2,68	2,19	1,013
DV70	RQR 5	34,8	2,83	2,57	1
DV80	RQR 6	37,8	2,99	3,01	0,993
DV90	RQR 7	41,0	3,18	3,48	0,988
DV100	RQR 8	44,2	3,36	3,96	0,986
DV120	RQR 9	50,8	3,73	5,00	0,986
DV150	RQR 10	61,2	4,38	6,55	1,002

Note: Note: These values are typical values measured at PTB in Germany in 2009.

- Piranha/RTI MAS-1 Probe, Invasive mAs probe (sensitivity 1 nC/mAs)

Module type	Range	Inaccuracy	Resolution
Tube charge	0.001 mAs –	±1 % (for time > 0.1 ms)	
Tube current	0.1 – 3000 mA	±1 % or ±10 µA (for time >100 ms)	Typ. noise:±1.5 µA
Pulse tube current	Lower limit 1 mA, otherwise same as tube current.		
Time ₁	0.1 ms – 34000 s 1 – 65535 pulses	±1 % or ±0.5 ms ±1 pulse	Resolution 0.5 ms

Note 1: When the Piranha internal detector is used simultaneously, the default mode of operation is to use the internal detector for time measurement.

Note 2: The tube current is calculated as the tube charge divided by the time. See also [Waveforms and Triggers](#) [47].

- Piranha/RTI MAS-2 Probe, Non-invasive mAs probe (sensitivity 1 nC/mAs)

Module type	Range	Inaccuracy	Resolution
Tube charge	0.1 mAs –	±5 % (for time > 0.1 ms)	
Tube current	10 – 4000 mA	±5 % or ±2 mA (for time > 20 ms) (±3 % at 250 mA)	Typ. noise: ±1 mA
Pulse tube current	Lower limit 50 mA, otherwise same as tube current.		
Time₁	0.1 ms – 34000 s 1 – 65535 pulses	±1 % or ±0.5 ms ±1 pulse	Resolution 0.5 ms

Note 1: See also note 1 and 2 for the Piranha/RTI MAS-1.

- Piranha/RTI Light Probe, Light detector (typical sensitivity 62.5 pA/(cd/m²) or 167 pA/lx)

Module type	Range	Inaccuracy	Resolution
Luminance	0.04 – 128 000 cd/m ²	±5 % or ±8 mcd/m ²	±0,5 mcd/m ²
Illuminance	0.014 – 48 000 lx	±5 % or ±3 mlx	±0.2 mlx

Note 1: It is recommended to keep the monitor adapter's inside and lux adapter's white surface clean from dust to meet the specifications. Use a lint-free cloth or blow air to remove any dust. Please see the Light Probe manual (available on the web or Product CD/USB) for detailed information on cleaning.

- CT-DP, CT Dose Profiler (typical sensitivity 3.6 µC/Gy)

Module type	Range	Inaccuracy	Resolution
Air kerma (Dose)	1,8 nGy – 22 kGy 200 nR – 2.5 MR	±5 % ±5 %	
Air kerma rate (Dose rate)	0.7 µGy/s – 2.2 Gy/s 77 µR/s – 260 R/s 4.6 mR/min – 15 kR/min 280 mR/h – 920 kR/h	±5 % or ±150 nGy/s ±5 % or ±16 µR/s ±5 % or ±1 mR/min ±5 % or ±60 mR/h	Typ. noise: ±5 nGy/s WF@2kSa/s: ±2 µGy/s
(Free-run or Timed)	67 nGy/s – 2.2 Gy/s	±5 % or ±15 nGy/s	

2.5 Standards and Compliances

Hereafter you can find declarations of conformity for the Piranha system.

2.5.1 Waste Electrical and Electronic Equipment (WEEE)

The European Union Directive 2002/96/EC on Waste from Electrical and Electronic Equipment (WEEE) places an obligation on manufacturers, distributors, and retailers to take back electronics products at the end of their useful life.

The WEEE directive covers all RTI products being sold into the European Union (EU) as of August 13, 2005. Manufacturers, distributors, and retailers are obliged to finance the cost of recovery from municipal collection points, reuse, and recycling of specified percentages per the WEEE requirements.

Instructions for disposal of WEEE by Users in the European Union



The symbol, shown left, is marked on the product, which indicates that this product must not be disposed of with other waste. Instead, it is the user's responsibility to dispose of the user's waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. The separate collection and recycling of waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information about where you can drop off your waste equipment for recycling, please contact your local distributor from whom you purchased the product.

2.5.2 Manufacturer's Declaration of Conformity

Declaration of Conformity

We, RTI Group AB, Flöjelbergsg. 8 C, SE-431 37 MÖLNDAL, Sweden,
declare under our sole responsibility that the product:

Product name:	Piranha
Product model number:	140 to 657
Type of equipment:	kV- & Electrometer
Intended use of this product:	See separate document no: 2-CE-9620000-0, "Intended use of the Barracuda, Piranha and Cobia"

is in conformity with the provisions of the following EC Directive(s):

- 2014/30/EU Electromagnetic Compatibility (EMC) Directive
- 2014/35/EU Low Voltage Directive (LVD)
- 2015/863/EU Restriction of Hazardous Substances in EEE Directive (RoHS 3)

and that the following standards and/or technical specifications referenced below
have been applied:

EN 55032/B, EN 55014, EN 55024, EN 60950-1, EN 61000-3-2, EN 61000-3-3, EN 61000-4-2, EN 61000-4-3, EN 61000-4-4, EN 61000-4-5, EN 61000-4-6, EN 61000-4-11, EN 62368-1 + AC1:2015, ETS 300 342-1



Möln dal, 2019-11-11

A handwritten signature in blue ink, appearing to read 'Fredrik Ljungberg', is written over a dotted line.

Fredrik Ljungberg, President

2.5.3 FCC Certification

Piranhas of product version 3.1 and newer contains FCC certified transmitter module (Bluetooth).

FCC ID R47F2M03GX

This device has been tested and found to comply with the limits for a Class-B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in commercial environment. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used according with the instruction manual, may cause harmful interference to radio communication. Operation of this equipment in a residential area is likely to cause harmful interference, in which case the user will be required to correct the interference at his own expense.

2.6 Maintenance


2.6.1 Updating the Piranha Firmware

All firmware that is controlling the function of the Piranha is stored in flash memory to allow quick and easy update. The RTI Updater with the latest firmware is always available free of charge on the RTI Web site at <http://www.rtigroup.com>. To update your Piranha you must first download the latest version and install it on a PC. The PC needs to have an USB port.


Note!

You will need to have access to an administrative account to install the software. To update the Piranha firmware (or bootloader):


1. First download the latest version of the RTI Updater Setup from RTI Web site (<http://www.rtigroup.com>).
2. Unzip the file and run the file "RTI Updater Setup.exe" to install it on your PC. In the end of the installation process you will get the question if you want to run that updater immediately. If you have your Piranha available you can connect it as described in step #3. Answer "Yes" and continue with step #6.
3. Connect the Piranha. Use the USB cable that came with your Piranha to connect your Piranha to one of the USB ports on the PC. Power on the Piranha. Use the power supply to ensure that no power failure occur during the update process. If you do not have a power supply available, make sure you have fresh batteries in the Piranha. You will get a notice about that.

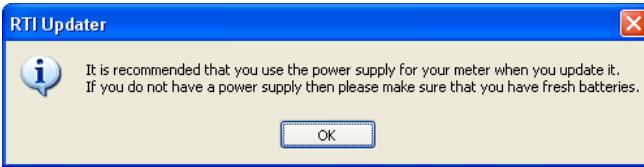


4. Go to **Start Menu | RTI Electronics | RTI Updater** and select the RTI Updater.
5. The RTI Updater starts and locates the Piranha automatically if it is connected to an USB port.

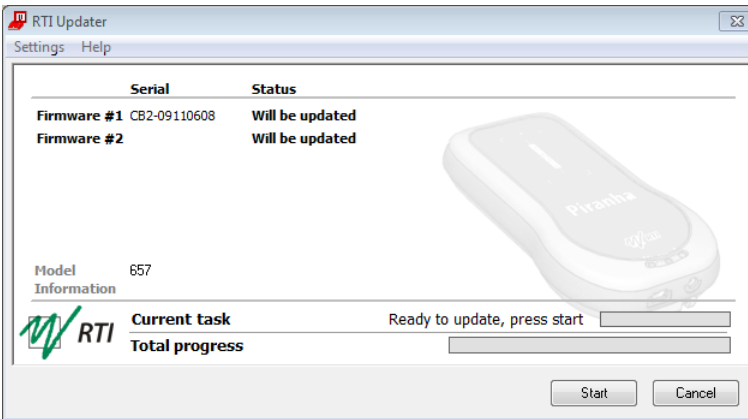

6. If the Piranha is found, the window in the figure above is shown. The different modules are checked and after a while the start button is enabled. Click **Start**. If the Piranha cannot be found, a message with suggested solutions is shown.
7. The updating process starts. The RTI Updater checks the current versions and compares with the update. Modules with old firmware are automatically updated.

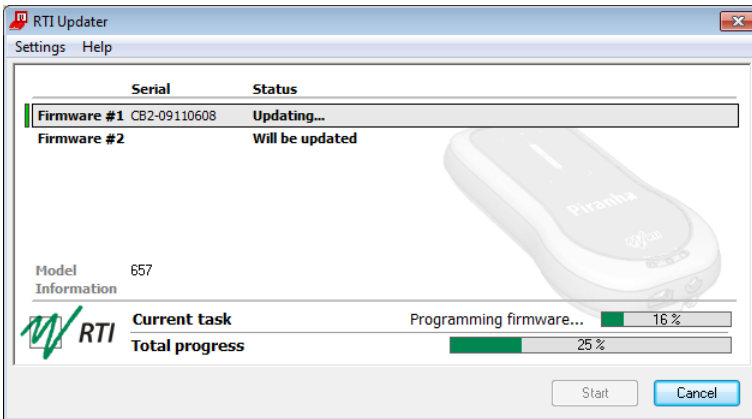


8. Note that storing the new firmware in the flash memory may take several minutes for each module. The RTI Updater will indicate which modules have been updated.
9. Power off the Piranha and disconnect the serial cable when the program indicates that everything is OK.

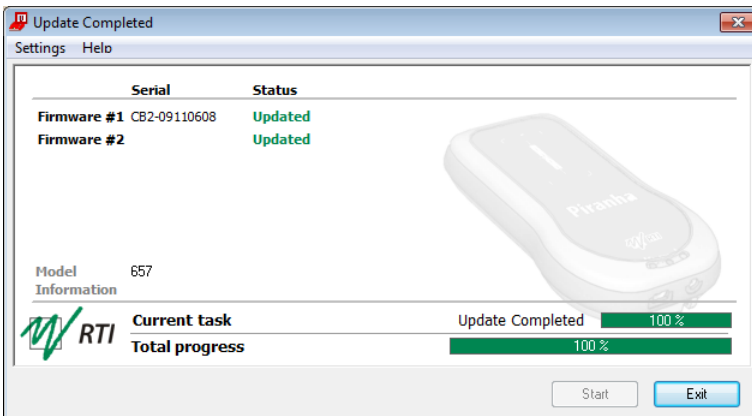

If you have any problem with your Piranha after updating, re-install the firmware again before contacting your local distributor or RTI. To re-install firmware repeat step 1 to 9 above, but before performing step #6 go to the menu **Settings** and select **Always Overwrite**.

Note!

If you want to see more details of what is updated, use the menu **Settings - Advanced**, and you will see more information as shown in the figure below.

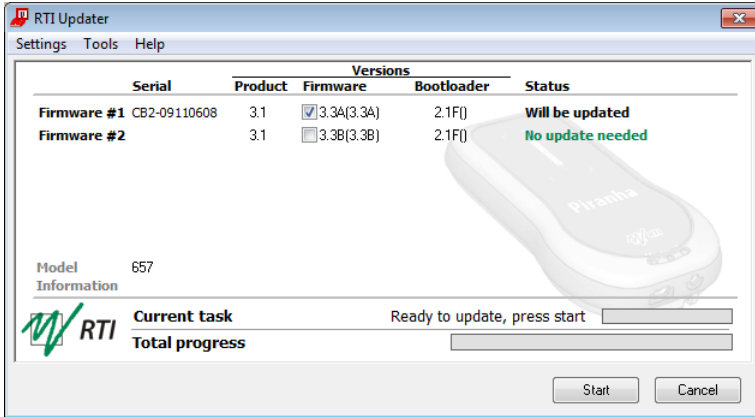


Note!

You normally also need to update the software, when you update the firmware. See the QABrowser manual and the Ocean manual for details.


2.6.2 Managing Detector Calibrations

RTI Detector Manager is a special Windows software that gives an overview of all calibrations for the detectors and probes in your system. You will find the RTI Detector Manager on your Product CD, in the folder **\Software\RTI Detector Manager**. Start the file **RTI Detector Manager.exe** by double-clicking it.

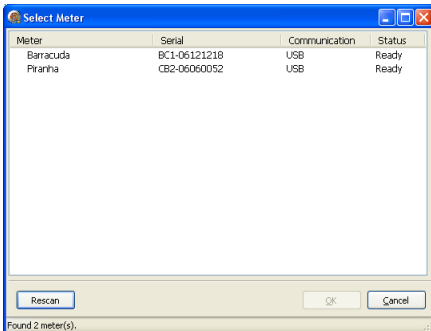


Select the instrument of interest (Piranha) and click **OK**.


If no instrument appears, check the communication cable and that the Piranha is powered on, then click **Rescan**.

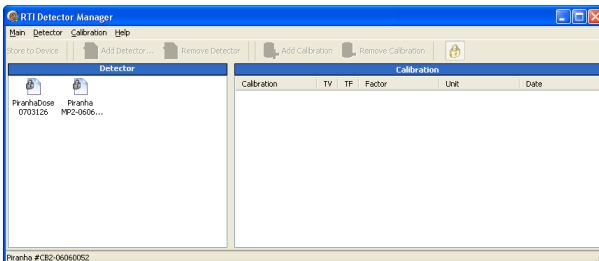

Next, the available detectors are shown to the left. The Internal detector is always available, but external probes will only show up if they are attached.

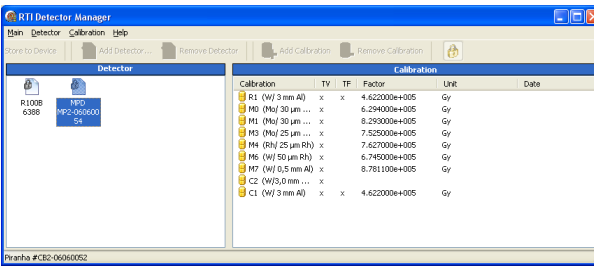

When clicking a detector, the available calibrations will show up to the right (In this case the Internal Detector is highlighted). The TV and TF columns show an **x** if there are calibrations for Tube Voltage and/or Total Filtration. The factor column shows the calibration factor (for dose in this case).

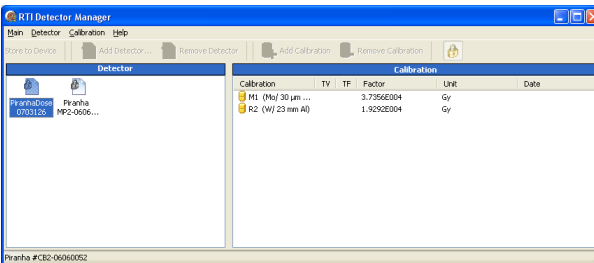

Here is another example (Piranha Dose Probe). This type of detector only contains a calibration factor for dose.

2.7 Measurement Settings

2.7.1 Settings - Conditions

Here general conditions for the measurements are shown. Different values can be shown depending on selected measured parameter.

Conditions - TF and Waveform

Total Filtr. Shows actual total filtration value. *Estimate* means that a new estimation will be performed at next exposure and the values will be displayed on screen.

Waveform The waveform types supported are:

- DC/HF
- Single Phase
- 3-Phase 6-Pulse
- 3-Phase 12-Pulse
- AMX-4
- Pulsed

Note! In Ocean using Quick-Check you need to select the generator waveform type for each measurement session. Default is *HF/Constant potential*.

AMX-4

The difficulties when measuring tube voltage on a GE AMX-4 is a well-known problem. Due to high kV ripple at a frequency of 2 kHz it is hard for most non-invasive kVp-meters to follow the kV waveform correctly.

This waveform type has an agreement with measurements made with the Keithley Triad System 37946C mobile filter pack (50-135 kV), which is the only filter package recommended by GE. According to GE, the use of the standard Keithley 37617C W-R filter pack (50-150 kV) is not good enough. The results have further been verified with measurements with a traceable high voltage divider that has sufficient bandwidth to accurately follow the kV ripple from the AMX-4.

Therefore it is important to select the **AMX-4** waveform (under **Meter Settings**). More about the AMX-4 correction can be found in the Application Note 1-AN-52020-1 from RTI Group AB.

Pulsed

This waveform type should be used for pulsed fluoroscopy especially when the pulses do not have a "good" square waveform shape. The exposure time must be longer than the selected recording time when using this waveform type. (Pulsed waveform type is selected under **Settings | Conditions** in the same way as the AMX-4 waveform type.)

Conditions - TP-factor

If an ion chamber is used, temperature and pressure can be specified to get correct dose measurements.

Note! The Internal detector and other semiconductor detectors are virtually not affected by temperature and pressure.

The **TP Factor** is calculated as follows:

$$TP = P_0/P \times T/T_0$$

where: P_0 and T_0 are the reference air pressure and absolute air temperature (normally $P_0=101.3$ kPa and $T_0=293$ K [20 °C])

T and P are the actual absolute air temperature and pressure in the same units (kPa and K)

Temperature This value is used to calculate the TP-factor when ion chambers are used. Temperature can be specified in Kelvin, degrees Celsius, or degrees Fahrenheit. Unit is chosen in Ocean under Options | Default Units.

Pressure This value is used to calculate the TP-factor when ion chambers are used. Air pressure can be specified in several different units. Unit is chosen in Ocean under Options | Default Units.

Please note that the pressure easily changes by 10 %. For the same effect from temperature, it must change 30 °C (or 54 °F).

Pulse rate

If a pulsed mode is used, like pulsed fluoroscopy or pulsed radiography (cine) the pulse rate can be specified in pulses per second (same as Hz). This allows you to get a dose/pulse reading even if the detector used (e.g. ion chamber) is too slow for the Piranha electrometer to detect the pulses. A solid-state detector, like the Piranha Dose Probe, is however fast enough to detect the frequency even for very low-level signals.

Compression paddle

For mammography, it is sometimes easier to do measurements with the compression paddle in the field. The compression paddle will however affect the Piranha kV and dose reading. With this setting, all the measured values (kV, dose and HVL) will be corrected according to what the user has selected.

When selected you will see the settings for *Scatter factor* and *Equivalent thickness*. The thickness is given in "mm Al". If you do not know, ask the manufacturer or make a comparison with Aluminium filters.

Scatter factor If an ion chamber is positioned just below the compression paddle, the measured dose will rise, because of side scattering from the compression paddle material. The effect of this is depending on the ion chambers angular dependence. Since the Piranha is almost insensitive to this, you can put a number here to compare readings from the Piranha with readings from an ion chamber. See also section [Average Glandular Dose, AGD \(MGD\)](#)^[60].

Equiv. thickness The given equivalent compression paddle thickness is used to increase the accuracy of dose measurements when dose is measured below the compression paddle. It is given in equivalent thickness of aluminium.

Note: This feature can also be used if you have additional filtration in the beam. Add the equivalent thickness of aluminium.

Beam Correction Sometimes you may want to make comparable measurements with known mechanical setup. For instance if you want to emulate ion chamber measurements in a particular scattering situation. Then you can set a *Beam Correction factor* to get that reading. In this case the ion chamber measures an extra 25 % from side and back-scatter. Using this factor makes the readings to be the same. It is of course important that the mechanical setup in these cases are the same.

2.7.2 Settings - Piranha

Here general measurement settings for the Piranha are shown.

Post Delay The post delay time defines how long time the Piranha shall wait and "look for more" after detecting what can be considered to be "the end of the exposure". Default value is 250 ms. The post-delay is necessary when measuring on units with some kind of pre-pulse or for pulsed exposures.

Trig source

This setting makes it possible to define the trig source for the electrometer module.

Available settings are:

- *Individually*, each detector starts to measure individually when it detects a signal.
- *Internal detector*, the measurement of all parameters (all modules) start when the Internal detector starts to measure.

Default value is always Internal detector when it is used. This is the recommended trig source.


Note! In Ocean Quick-Check this setting is not available.

Trig level (time)

Here you can set the level used for irradiation time measurements. "Trig level (time)" (TL) is normally set to 50 % of the peak waveform (S_{PEAK}), but can be set between 10 and 90 %. The irradiation time is then calculated as the end time minus the start time.

- The start time is the first time the signal goes above $TL \times S_{PEAK}$.
- The end time is the last time the signal goes below $TL \times S_{PEAK}$.

See example below.

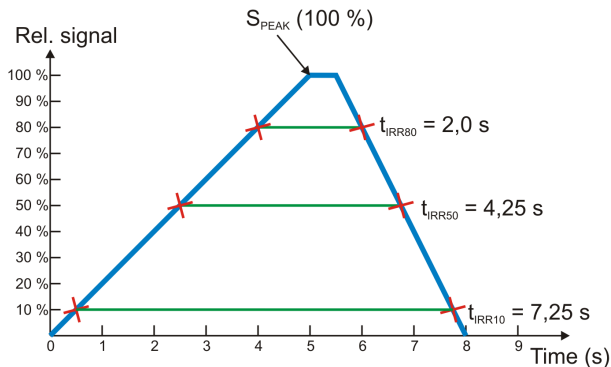**Measuring Mode**

This setting defines when Piranha shall send measured values to Ocean.

Three different alternatives are available:

- *Normal*, the Piranha is continuously sending data as long as radiation is detected. Values to the display/software are updated about every two seconds. Typically used for general measurements.
- *Timed*, the user sets a measurement time. The user then starts the measurement and the Piranha will measure all radiation received during the

measurement time, without any trig levels. When the time has passed, a reading will be presented.

- *Free run*, the Piranha will continuously measure the radiation without any trig levels.

Default value is set according to selected type of measurement and this parameter normally never needs to be manually changed, unless really low-level measurements are to be accomplished.

**Waveform
rec. time**

The Piranha can sample a maximum of 2048 samples. Ocean supports the full depth, but the QABrowser is only able to show a total of 640 samples. The sampling interval is normally 0.5 ms, giving a total measurement window of 1024 ms in Ocean and 320 ms when using the QABrowser. By increasing the sampling interval, a much longer sampling window can however be selected. This is very handy when longer exposure times are used and the waveforms need to be viewed.

Start after delay

When this is selected, the waveform recoding will start after the set delay. This can be useful if you want to study a phenomenon that occurs after the normal waveform recording time. When this is selected the electrometer waveform will not show simultaneously and you will get a warning that the irradiation time measurement is inaccurate. The reason for this is that the Piranha needs the waveform from start to be able to accurately calculate the irradiation time.

2.7.3 Settings - Internal detector

Here general measurement settings for the Internal detector are shown. You can find information about the different parameters below.

**Sensitivity
Dose/TF**

This is used to set the dose and TF sensitivity for the Internal detector.

The sensitivity can be set to: Low, High, and Very High.

Sensitivity kV

This is used to set the kV sensitivity for the Internal detector.

The sensitivity can be set to: Low and High.

Delay

The delay time defines how long time the Piranha shall wait before starting to measure kVp after that radiation has been detected.

Window

This gives the possibility to define a fixed time that Piranha measures kVp after that the delay time has expired. Default value is always "Infinite".

2.7.4 Settings - Other Detectors

- Sensitivity** This is used to set the sensitivity for the electrometer module. The sensitivity can be set to: *Low* and *High*.
- Threshold** This is used to set the trig level. It can be set to *Low* ($\frac{1}{2}\times$), *Normal*, $2\times$, $4\times$, and $8\times$. The default value is "Normal". The setting "Low" can be used if low signals are measured and a lower trig level is required. However, the risk for false triggering increases when "Low" is used. To avoid false triggering in a noisy environment use one of the "higher" threshold levels.
- Note!** In Ocean Quick-Check this setting is not available.
- Normalized exposure** Note: Only for dose detectors!
If *Normalized exposure* is checked, you have the option to normalize the dose reading to any given distance. Here you can enter your Source to Detector Distance (SDD) and a normalizing distance (SDD Norm), that you want the dose normalized to.

Chapter 3

Measurement Principles & Theory

3 Measurement Principles & Theory

The Ocean software has a number of measurement algorithms and applications built-in. This section describes some about the principles, how some values are calculated, and the basic use of such measurements.

3.1 Overview of Capability for Measurement Modes

The following graph shows an overview of some common capabilities the different X-ray measurement types have in the Piranha.

Modality	Estimated TF	Quick-HVL
Radiography	OK	OK
Cine/Pulsed exposure	OK	OK
Fluoroscopy	OK	OK
Pulsed Fluoroscopy	OK	OK
Mammography	–	OK
CT	OK	OK
Dental	OK	OK
Panoramic Dental (OPG)	OK	OK

3.2 Update Modes

The following three update modes are available:

- **Normal**, the Piranha is continuously sending data as long as radiation is detected. Displays in the PC are updated about every seconds. Typically used for most circumstances. Reset time is one second.
- **Timed**, the user sets a measurement time. The user then starts the measurement and the Piranha will measure all radiation received during the measurement time, without any trig levels or background compensation. When the time has passed, a reading will be presented. It has a long reset time for increased accuracy, which varies with the sensitivity, as seen in the table below.
- **Free run**, the Piranha will continuously measure the radiation without any trig levels or background compensations. No applications are available when using this mode. The mode has a feature called moving average which calculates the average of the measured values during a defined time, to increase accuracy by lowering the time resolution. This function gives a larger stability to the measurements. Free run also has a long reset time for increased accuracy, see the table below.

Default value is set according to selected type of measurement and this parameter normally never needs to be manually changed, unless really low-level measurements are to be accomplished.

However, to measure on real low-level signals the *Timed* or *Free run* update mode may be used.

Update mode	Auto reset	Reset time (s)	Sample time (ms)	Min. WF rec. time (s)	Max. WF rec. time (s)
Normal	Yes	1	0.5-64	1	524
Free run (Low/High sens.)	–	4	0.5-64	1	524
Free run (Very High sens.)	–	30	0.5-64	1	524

Note: The *Sample time* is the "resolution" of the waveform, i.e. time between two samples.

Auto reset means that a reset is performed after each trig off.

Reset time is the time it takes to perform a reset each time you hit *Reset*.

Sample time is the time between individual data point of the waveform.

Waveform recording time is the range of user selectable recording times the Piranha allows.

Note that in *Timed* and *Free run* you may get negative readings, for instance if you press reset when a signal is present on the detector.

3.2.1 Using Timed Update Mode

The Timed mode can be very useful both for very low dose rate measurements as well as for long duration measurements.

- For extreme low-level dose measurements you can improve your reading by subtracting the background level. First do a Timed measurement without exposing the detector to radiation and then do the same with radiation. The timed mode will use the same measuring time and the first reading can be subtracted from the first. Just make sure not to do a **Reset** between these measurements, as the **Reset** will also make an offset adjustment. Note also that low-level readings may give inaccurate kV readings.
- For long duration measurements, cases with slowly rising and falling output, or cases with very low pulse rate, timed mode may also be useful. For instance on CT machines where the rotation cannot be stopped.

3.2.2 Using Free Run Update Mode

Free run update mode works almost exactly as the ordinary *Continuous* update mode. There are however two differences:

1. Since there is no trig level, you will be able to measure lower, but there will be no time reading unless the signal goes over the trig level.
2. You can select a moving average function. This lets you set a time for moving average, this time acts as a averaging window, moving through time.

Moving average

This function is intended for low level dose rate measurements where increased sensitivity and stability is needed. The function uses a moving average algorithm where the number of seconds is selected by the user.

During the reset process the user must make sure that the detector is not exposed to radiation. After the reset procedure the Piranha will start to show a value calculated as

the sum of the last X values divided by X (X is the number of seconds chosen by the user). For each new second that passes the last value in the stack will be discarded and a new value added. This means of course that it will take X seconds before the Piranha starts to show a valid value when the detector attached is exposed to a steady radiation level. In the same way it will take X seconds for the Piranha to show a zero value after the radiation has ended. Great care must be taken into choosing a time constant fitted to the nature of the signal.

Example

If you set the time to 8 seconds, each reading, will be the mean of the reading of the last 8 seconds. This means that it will take 8 seconds until the reading reaches a started set radiation level.

3.3 Display Messages and Active Messages

Even though the range of the Piranha measurement system is quite wide, sometimes the signal may get too low or too high. To inform you of this, there are display messages. These are mainly of two types, Active or Passive. Active messages are shown when the hardware settings can be adjusted to adapt the measurement ranges. The active message will just inform you that it is making an automatic adjustment and you can simply do another exposure/measurement. The active messages can be disabled, see the following section.

The passive display messages indicate what the problem is and possible remedies for them. These will show if there are no active messages, the active messages are disabled, or when no more automatic adjustment can be done.

3.3.1 Active Messages

In some rare occasions it might be helpful to disable the active messages, for instance if the detector signal is very noisy or there are pre-pulses that makes the system auto-adjust erroneously.

Below the various active messages are shown. Make sure to follow the text shown, since reset may be performed automatically. Otherwise do a **Reset** again.

High signal	One or several detectors have too high signal.
Low signal	The Piranha detects a signal but it is too low to present a reliable result.
High kVp	Measured tube voltage is higher than that of the selected kV-range.
Low kVp	Measured tube voltage is lower than that of the selected kV-range.

3.3.2 Display Messages

High signal

One or several detectors have too high signal.

- Lower the set sensitivity under settings.
- Reduce the mA and/or increase the distance from tube to detector.

Exp. < Delay

The exposure time is too short compared to the delay time.

- Increase the exposure time and/or reduce the values of delay and/or window time.

Keep in mind that for Ocean, some settings are automatically set by the type of measurement chosen.

High kVp

Measured tube voltage is higher than that of the selected kV-range.

- Change to a higher kV-range.

Low kVp

Measured tube voltage is lower than that of the selected kV-range.

- Change to a lower kV-range.

Low Signal

The Piranha detects a signal but it is too low to present a reliable result.

- Increase the mA and/or decrease the distance from X-ray tube to detector or change the sensitivity for the dose parameter to *High sensitivity* or even *Very High sensitivity*. Also the *kV sensitivity* can be changed. You find these settings under Meter Settings.

Reposition Detector

The radiation signal on D2 and D1 is not within 5 % (quota not between 0.95 to 1.05).

The most common reason for this is that the detector area is only partially irradiated, the detector is tilted, or the filtration differs between D2 and D1 (e.g. heel effect).

- Change the field size or move the detector into the central beam.


Negative Signal

The electrometer module detects a negative signal.

- Most common is that the mAs-probe have been connected in the opposite direction on the HV cable. Change the polarity of the current probe.
- Also small negative drift created from the detector source, typically initially after reset can give this message. Special care should be taken when using ionization chambers, since it may take a few minutes before the initial drift of the electrometer and detector has stabilized.
- Do a **Reset** to clear the message.
- This message does not appear in the *Timed* and *Free run* update modes.

3.4 Waveforms and Triggers

To get an understanding of how triggers, delays, and windows work, take a look at the waveform below. This is what happens during a standard exposure:

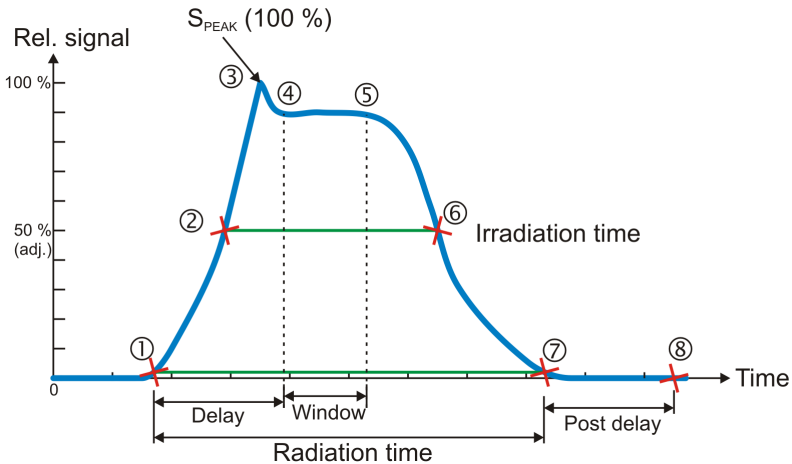

1. The radiation starts, i.e. it goes over the detector's lowest trig level.
2. The signal reaches 50 % of its maximum. This is the starting point for the irradiation time calculation. (The level is user adjustable.)
3. The signal reaches its maximum.
4. The *Delay* time is reached. (User adjustable.) kV integration window starts.
5. The *Delay+Window* time is reached. (User adjustable.) kV integration window stops.
6. The signal goes below 50 % of its maximum. This is the end point for the irradiation time calculation. (The level is user adjustable.)
7. The radiation ends, i.e. it goes under the detector's lowest trig level.
8. If the signal has been below the trig level during all of *Post delay*, the exposure is considered finished. All exposure readings are calculated.

Integrated signal (dose, mAs, etc.)

Is the integration of all signal which means the area below the curve above from point 1 to 7. During the measurement (exposure) the accumulated signal (dose, etc) is displayed where applicable.

Signal rate (dose rate, tube current, etc.)

During the measurement the mean signal for the last second is displayed. When the measurement (exposure) is over, point 8 above, the mean signal for the whole measurement is displayed. This signal rate is calculated as all integrated signal (as described above) divided by the irradiation time. If no irradiation time is possible to calculate, the radiation time is used instead.

This means that for long measurements you may see a change in the rate value (dose rate, etc) when the measurement is finished, if the signal level was changed during the measurement.

3.5 Measurement Principle for the Piranha

The following are the key features of the Piranha design:

- Small size
- Optimized filter packages for five different kV ranges
- Very sensitive and wide dynamic range
- Check filter for measurement geometry verification
- Single exposure estimation of total filtration and Quick-HVL
- Single exposure estimation of generator waveform type

The design of the detector package is very important to be able to measure kV and dose correctly in the whole range of 20 to 155 kV.

The Piranha design makes it possible to measure small field sizes, less than 3 mm width, and low output levels down to approximately 1 $\mu\text{Gy/s}$. Basically the detector packages consist of four separate electrometer channels connected to detectors D1, D2, D3, and D4 and a moveable filter package that can change to one of six positions, each a combination of different filters for the detectors. One of these positions is used as a "check-filter". It has the same filter thicknesses for both D1 and D2. When the detector is perfectly positioned and both detectors have the same radiation the ratio between the two signals should thus be exactly "1.000". This is very useful information, and testing this makes sure that your measurement geometry is fine, giving reproducible readings. The other 5 filter pairs have different thicknesses all optimized for different ranges of the tube voltage; two (1 and 2) are used for the low mammography energy range 20 to 45 kV, and three filters (3 - 5) are used for the radiography range 35 to 155 kV (35 - 75, 55 - 105, and 80 - 155 kV).

Using these four signals S1-S4 (from detectors D1 to D4) the Piranha can accurately calculate the corresponding tube voltage. The signal S3 is not affected by the moveable filters and is designed to measure the dose. This detector is marked by a square inside the rectangular detector area on the top panel. The reference depth for the sensitive area of the dose detector is 10 mm under the Piranha top panel surface.

The detector D4 is placed directly under D3 with additional filter in between. The ratio between S3 and S4 is used to estimate the total filtration for the radiography range. Using these signals together more accurate dose and tube voltage readings can be obtained.

Since all signals is measured simultaneously and with a relative high speed, the Piranha can thus automatically compensate the kV and dose for the dependence of the waveform and inherent/added tube filtration.

Chapter 4

Measurements with the Piranha System

4 Measurements with the Piranha System

4.1 Introduction

The Piranha system can, depending on model, measure up to eight parameters simultaneously plus three waveforms from a single exposure:

- kVp
- Dose and dose rate
- Exposure time
- HVL
- Estimated total filtration and determined waveform type
- mAs and mA
- pulses
- kV waveform
- Dose rate waveform
- mA waveform
- various pulsed fluoroscopy parameters

4.2 Radiography

Measuring the kVp on a radiographic units is straightforward since the Piranha can automatically detect and compensate for variation in the radiation quality. It is also easy to check that the detector area is fully and uniformly irradiated. Practically this means that the kVp value can be measured in the range 1.0 to 50 mm of total filtration.

Therefore the Piranha can be placed in the beam wherever you want, as long as it passes the *Position Check*. It also has a very wide dynamic range so it very rarely happens that the signal level is not enough to get a correct kVp value. The radiography kV range is 35 to 155 kV. The displayed dose value has no energy dependence since it is automatically compensated for each exposure since both the kV, estimated filtration, and the waveform are measured.

4.2.1 kVp, Time, Dose, and Dose Rate

To measure on radiographic units:

1. Set up the Piranha and the handheld computer according to the description in [Setting Up the Piranha](#)^[10].
2. Place the detector on the table at the distance that is clinically relevant.
3. It is recommended to make a check measurement at 70 kV to confirm that the detector area is uniformly radiated.
The Piranha automatically changes back to the previous selected kV range.
4. Set kVp and mAs (or mA/time) to the desired values.
5. Make an exposure.

6. Read the values.

4.2.2 Dose Measurements with Piranha Dose Probe

1. Place the Piranha Dose Probe in the field and connect the cable to the Piranha input.
2. Set kVp and mAs (or mA/time) to the desired values.
3. Make an exposure.
4. Read the values.

4.2.3 Quick-HVL and Total Filtration

Total filtration

The estimated Total filtration is measured with the Piranha internal detector, in one exposure, simultaneously with all other parameters.

The estimated Total filtration is also in use for internal compensation of kV and dose values.

Quick-HVL

The Quick-HVL is measured with the Piranha internal detector, in one exposure, simultaneously with all other parameters.

4.3 Cine/Pulsed Radiography

For under-table cine measurement turn the Piranha upside-down.

The *Position Check* should be used to confirm the position. To be able to protect the image intensifier from the relative high output cine pulses a lead apron can be placed over the image intensifier input screen. The Piranha automatically measures the number of pulses based on information from the radiation waveform. It uses a 50 % trig level based on the maximum signal level.

4.3.1 kVp, Time, Dose, and Dose Rate

Use the same procedure as for the normal radiography measurement.

Note that if it is difficult to get a good pulse rate reading, you may use a manual pulse rate setting, as described under [Measurement Settings](#) ^[35] to get a dose per pulse reading.

4.3.2 Pulse Measurements with Piranha Dose Probe

Use the same procedure as for the normal radiography measurement. Note that if it is difficult to get a good pulse rate reading, you may use a manual pulse rate setting, as described under [Measurement Settings](#) ^[35] to get a dose per pulse reading.

4.3.3 HVL, Quick-HVL, and Total Filtration

Use the same procedure as for the normal radiography measurement.

Note that if it is difficult to get a good pulse rate reading, you may use a manual pulse rate setting, as described under [Measurement Settings](#) [35] to get a dose per pulse reading.

4.4 Fluoroscopy and Pulsed Fluoroscopy

For under-table fluoro measurement turn the Piranha upside-down. Use the optional detector rod to be able to put the detector in the cassette holder or on the image intensifier without risk of hazardous X-ray.



The *Position Check* should be used to confirm the position of the Piranha. When you select fluoroscopy or pulsed fluoroscopy (as type of measurement) the Piranha system automatically changes to continuously updating the display and using the highest possible sensitivity.

If dose rate or image intensifier dose rate is selected the Piranha or external Dose Probe can be used.

Normally the external Dose Probe is used to be able to measure the lowest possible dose rate levels down to 0.1 $\mu\text{Gy/s}$. Another reason to use the external Dose Probe is that the detector is much smaller than the Piranha making it easier to position in front of the image intensifier without affecting the measuring field for the mA feedback loop.

If the image intensifier manually can control the mA and kV, then you can use the Piranha for measurements down to about 0.7 $\mu\text{Gy/s}$. For pulsed fluoroscopy even lower levels can be measured.

As a secondary parameter the total dose is accumulated. After you have turned off the fluoroscopy unit, this value is used to calculate the average dose rate as total dose divided by the exposure time.

Note that for very low dose rate values the exposure time cannot be measured accurately and the last dose rate value cannot be stored automatically in the display. Then tap **Hold** to "freeze" the current value in the display. The waveform is also acquired when you tap **Hold**. Waveform is also automatically acquired when the selected delay time expires.

Select "I.I. input dose rate" as measuring parameter to be able to measure lowest possible dose rate and tap **Hold** to "freeze" current value in the display. The total accumulated dose is showed after you have switched off the fluoroscopy unit.

4.4.1 Image Intensifier Input Dose Rate

Use patient equivalent phantom to measure the image intensifier input dose rate according to manufacturer's specification:

1. Place the Piranha Dose Probe in front of the image intensifier but outside the measuring field for the mA feedback loop. You may use the optional detector rod that can be attached to Piranha Dose Probe to position the detector without risk for hazardous X-ray exposure. Observe the image on the monitor.

Note! Since the external dose probe (Piranha Dose Probe) is not sensitive to back scatter, a lower value compared to a transmission ion chamber is typically detected (typically in the range of 5 - 20 %).

Tip! You may use the *beam correction factor* to make automatic corrections. The *beam correction factor* may also be stored permanently in a *Favourite* for easy access.

2. Start the fluoroscopy.
3. The image intensifier input dose rate is measured and the display is updated approximately every four seconds. Click **Hold** to "freeze" the currently shown value in the display. The waveform is also acquired when Hold is activated. Release Pause/Hold, the display will continue updating the display continuously.
4. Stop the fluoroscopy.
5. Read the values.

4.4.2 Pulsed Fluoroscopy

Select type of measurement in similar way as for normal continuously fluoroscopy to setup the system.

Piranha is set-up for this application to calculate the number of pulses per second (Hz or pps), the dose/pulse, and pulse dose rate. The picture below explain the difference between the pulse dose rate and the traditional dose rate.



(Note that for DC waveform, pulse dose rate and dose rate gives the same value.)

When measuring tube voltage on pulsed fluoroscopy there is an additional waveform type available. This waveform type is called *pulsed* and is recommended for pulsed fluoroscopy measurements, especially if the pulses are not square wave shaped, since this can result in low tube voltage readings.

4.5 Mammography

This topic will describe how to measure kVp, dose, HVL, AGD (MGD), and time on a mammography unit using the Piranha.

4.5.1 General

To measure kVp on a mammography unit is straightforward. This is true since the Piranha automatically can detect whether the detector area is not fully uniformly irradiated, by means of the *Position Check*. The mammography kV calibrations available for the Piranha is ranging from 18 to 49 kV.

Note! To be able to trust the kVp reading it is always very important to make the *Position Check* to verify that the whole detector area is uniformly irradiated. The *Position Check* is normally started automatically every time you change *Radiation Quality*, but please make sure to do a *Position Check* every time the Piranha is repositioned.

The kV and radiation waveform is always stored together with the RTD values and can be displayed by tapping the **Wave** button. The kVp calibration for Piranha is made without the compression paddle in place.

The purpose of dose measurement is often to determine the ESAK, *Entrance Surface Air Kerma* (or ESE, *Entrance Skin Exposure*).

It is recommended to perform dose measurements according to a mammography protocol. One is the "European Protocol on dosimetry in mammography EUR 16263 EN from the European commission". Chapter 3 in this protocol describes in detail the determination of AGD, *Average Glandular Dose* (or MGD, *Mean Glandular Dose*). The AGD is derived from measurements of the HVL and of the ESAK. Make use of tabulated conversion factors from ESAK to AGD. See [Average Glandular Dose, AGD \(MGD\)](#).

4.5.2 Setting Up the Piranha for Mammography



To set up the Piranha:

1. Power on the Piranha using the power switch. Optionally you may connect the power supply from the power outlet to the USB port.
2. Make sure that the image receptor is positioned at a clinically relevant distance (typically 600 mm).
3. You should place the Piranha flat on the image receptor with its long axis parallel to the chest wall making sure the centre of the detector surface is placed in the centre of the light field, as shown in the pictures above (40 mm distance shown). This placement of the Piranha makes the detector surface perpendicular to the cathode/anode axis, to avoid influence from the heel effect.

Note! For general mammography, it is important that the USB port points in the patients left direction, as shown in picture.

Tip! To be able to get comparable results, please consider the position of the Piranha. The Piranha should be placed at a clinically relevant distance from the

chest wall. Recommendations for this varies, typically between 40 and 60 mm. For Europe, 60 mm is the recommended distance (Ref. ECR 16263 EU).

4. Connect the devices.

Handheld: For Bluetooth (wireless) nothing is needed.

PC: connect the USB cable. For Bluetooth (wireless) attach the Bluetooth adapter to the PC (if not built-in).

5. Power on the handheld computer or the PC.

Now everything is set up with the hardware. Please continue in one of the following sections, depending on what you want to measure.

4.5.3 kVp, Time, and Dose Measurements with the Internal detector

Set up the Piranha and the handheld computer according to the description in [Setting Up the Piranha for Mammography](#)^[55].


1. Select Type of Measurement and Parameter, as shown below.
2. Select the correct radiation (beam) quality.
3. Make a *Position Check*, as shown above. It is recommended to make the check at 28 kV. After the check the Piranha automatically changes back to the previously selected kV range.
4. Set kVp and mAs (or mA/time) to the desired values.
5. Make an exposure. The RTI logo flashes to indicate that the Piranha has detected the exposure.
6. Read the values.

Note! There are some mammographic units that are bit peculiar when it comes to kV measurements, for instance The Hologic Selenia and IMS Giotto. In those cases, RTI have updated Application Notes, and there may be some even for other units. Please check the RTI website (www.rti-group.com) for the latest info. For Sectra MDM, Fischer Senoscan and other scanning beam units, please see the section [Scanning Beam Mammography](#)^[67].

4.5.4 Dose Measurements with the Piranha Dose Probe

Tip! It is often more convenient to use the Piranha to measure dose for a mammography tube since no manual energy compensation has to be done, as is the case with the Piranha Dose Probe (or TP compensation for an ionization chamber).

Measuring procedure

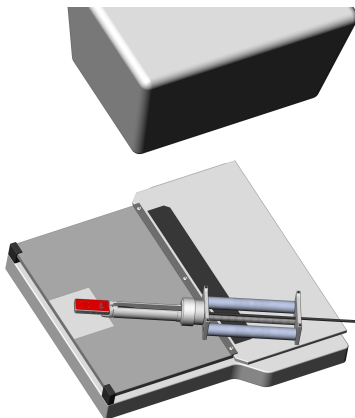

1. Place the Piranha Dose Probe in the field and connect the cable to the Piranha input.
2. Set up the Piranha according to the description in [Setting Up the Piranha for Mammography](#)⁵⁵.
3. Follow the same step as for the measurements with Piranha but select **Dose** as parameter. You will also need to select the External detector.

4. Select radiation quality from the detector list.
5. Set kVp and mAs (or mA/time) to desired values.
6. Make an exposure.
7. Read the dose value. The dose reading has to be corrected manually according to the Piranha Dose Probe DETECTOR DATA manual, unless set values for kV is used in Ocean.



You may store the correction as a *Beam Correction Factor* in a *Favourite* for a specific kV, to do the correction automatically.



To get a good HVL value, using the Piranha Dose Probe, you must correct it according to the tables in the Piranha Dose Probe DETECTOR DATA manual. It is often more convenient to measure HVL with the Piranha using the built-in HVL application.

4.5.5 Quick-HVL

Quick-HVL

The Piranha is able to measure HVL for mammography in one shot. Quick-HVL is available for measurements with or without compression paddle.

4.5.6 Mammo Compensations and Corrections

Here various corrections and compensations are described, that are of special importance for mammography.

4.5.6.1 Corrections for the Compression Paddle

The Piranha is well collimated above its small detector area, and will measure the same whether the compression paddle is placed directly on top of the Piranha or high above. This is NOT true for an ion chamber.

A factor has been introduced which enables the Piranha to take the scattered radiation into consideration and produce measurement results as if it was an ion chamber which senses the scattered radiation directly.

When an ion chamber is placed directly below the compression paddle, a relatively constant scatter factor of 6 % is found. The factor is typical for ion chambers such as Radcal 6M, PTW N23344, and Standard Imaging Magna 1cc.

When using Ocean, a 0.10 or 0.12 mm Al equivalent compression paddle is used. That is equal to approximately 3 mm of plexiglass (PMMA). Please see the Ocean manual for details.

Note! When comparing with typical mammographic ion chambers listed above, you should multiply the Piranha dose value with a scatter factor of 1.06 to make it measure as an ion chamber directly below the compression paddle.

More info about the correction for compression paddle can be found in Application Note 1-AN-52020-2 from RTI Group AB. Please also see section [Angular Sensitivity Piranha](#) ^[23] for details on sensitivity in different directions.

4.5.6.2 Normalization

A normalization function is available which enables all measurements to be virtually performed at the same distance, increasing productivity. According to European protocol (ECR 16263 EU, 1996), ESAK should be measured 45 mm above the breast support.

A practical consequence of usage of the normalizing function and scatter factor is that the Piranha can be kept at the same position on the breast support all the time when data is collected for AGD.

For an ion chamber it is not quite as easy because of the scatter contribution that is not allowed during HVL measurement. The ion chamber and/or the compression paddle must be moved to support good geometry.

More info about the correction and normalization function can be found in Application Note 1-AN-52020-2 from RTI Group AB.

4.5.6.3 Beam Correction Factor

Sometimes you may want to make comparable measurements with a known mechanical setup.

Assume that you want to emulate ion chamber measurements in a particular scattering situation. Then you can set a *Beam Correction factor* to get that reading like you used to. In this case the ion chamber measures an extra 3 % from side and back-scatter. Using this factor makes the readings to be the same. It is of course important that the mechanical setup in these cases are the same.

Note! You may use the *Beam Correction factor* to make compensations and corrections of various nature. Examples might be: energy corrections, angular corrections, field inhomogeneity corrections, etc. If you save this setting as a *Favourite*, you can have a quick way of repeatedly making a special measurement without any manual corrections.

4.5.6.4 Corrections for Angular Sensitivity

For mammography, the following correction table may be used at a SDD of 60 cm, if the Piranha is placed flat on the breast support. (This assuming that the focal point is situated at the chest wall, which normally is the case.)

You can find the product version on the label on the back side of your Piranha.

Distance from chest wall (cm)	Correction for Piranha v1.X (%)	Correction for Piranha v2.X and higher (%)
0	0	0
1	+1.9	+0.01
2	+3.8	+0.06
4	+7.6	+0.22
5	+9.5	+0.35
6	+11.5	+0.50
8	+15.3	+0.88
10	+19.1	+1.38

Note! Rule of thumb for v1.X: add 2 % per centimeter from the chest wall at 60 cm SDD, i.e. for 4 cm use +8 % correction.
As seen for product version 2.X+, no correction is necessary.

Tip! You may use the *Beam Correction Factor* together with *Favourites* in Ocean or QABrowser to automatically do a specific correction. See [Beam Correction Factor](#) ^[59].

See also [Angular Sensitivity, Piranha](#) ^[23].

4.5.7 Average Glandular Dose, AGD (MGD)

The AGD (average glandular dose) is derived from measurements of the HVL and of the ESAK, entrance surface kerma (or ESE) making use of tabulated conversion factors from ESAK (or ESE) to AGD (or MGD). The tabulated data has been derived from Monte Carlo calculations and has been verified experimentally.

To determine the AGD a standard phantom should also be used when the ESAK (or ESE) value is measured with the Piranha.

Correct measurement of the Average Glandular Dose (AGD) with the Piranha

In most situations you can perform measurements for a mammographic unit with the Piranha instead of a dedicated ion chamber. Since the Piranha compensates for energy dependence, the readings are in direct comparison with readings from a reference class ion chamber. When measuring the AGD you should always have the compression paddle in place.

Important quantities to measure

The most common measurements for a mammographic system are conducted to determine the average glandular dose (AGD). The AGD values are based on measurements of ESAK (entrance surface air kerma) and HVL. To do the measurements correctly and according to standards, the radiation detectors should be placed directly below the compression paddle. This introduces extra scattered radiation due to the compression paddle which is important to include when determining ESAK. On the other hand, the HVL measurement should be done without any scatter contribution and with good geometry.

HVL

The Piranha is well collimated above its small detector area. Due to this fact it registers a narrower angle of the X-ray field and thus much less scattered radiation compared to an ion chamber. It has built-in good geometry and is therefore ideal for HVL measurements. Hence, the HVL filter can be placed on top of the compression paddle without any extra collimation even at close distance to the Piranha. The Piranha has a built-in HVL application which should be used to get accurate HVL readings.

4.5.8 Mammographic Pre-pulses

Some mammography systems (e.g. GE DMR system) use a pre-pulse to determine what radiation quality to use for a specific patient.

The time elapsing between the pre-pulse and the real exposure is usually about one second. Therefore the default post-delay of 250 ms will not cover both the pre-pulse and the real exposure. To get an overview of the signal output, set the post-delay to at least 1 s and the waveform recording time to a corresponding time. It is important to cover both signals. In this measurement setup, the Piranha will add the dose from both pulses. This is OK if the radiation quality is not changed between the signals.

If the Mammography unit changes the radiation quality after the pre-pulse however, the kV and dose is affected and the pulses should be treated separately. To collect data from real exposure, set the delay (not the post-delay) to exclude the pre-pulse.

4.5.9 Scanning Beam Mammography

When measuring on scanning beam mammographic equipment, like for instance Sectra MDM or Fischer Senoscan, two factors are very important.

1. You should place the Piranha flat on the image receptor. Then position it as described in section [Setting Up the Piranha for Mammography](#)^[55].
2. Always perform a position check. This makes sure than any field imbalances are corrected for.
3. If you use the compression paddle, make sure that you use the correct settings, see section [Corrections for the Compression Paddle](#)^[58].

Please also see the CT section for measurement tips on scanning beams. For Sectra L30, see special application note on the RTI website (www.rtigroup.com).

4.6 Dental and Panoramic Dental

This topic will describe how to measure kVp, dose, and time for a Dental and Panoramic Dental X-ray units using the Piranha only.


Set up the Piranha and the handheld computer according to the description in [Setting Up the Piranha](#)^[10]. To measure kVp for a dental unit is similar to measuring for a radiography units with the difference that the output level is much lower and the total filtration is normally around 2 mm Al.

The setup is straightforward and also to get the measured value. Most dental units is still single phase self-rectified and has 100 % radiation and kV ripple. In the case of one-phase dental units it is common that only the exposure time can be changed. In most cases the set tube voltage and current is fixed to about 65 kVp and 8 mA. A challenge can exist how to find a definition what measured value should be used. Furthermore the radiation output and the kV waveform are not stable for the first 200 ms or so, because that the tube filament current in most cases is not regulated.

The definition of both what is the true kVp and exposure time cannot be as easily determined without study the waveform and select appropriate measurement parameters as delay and window. Introducing small CCD detectors instead of film also demands carefully calibration of single phase dental systems. The tube voltage waveform is collected from 200 ms after start trig and the kVp is calculated based on the measuring window equal the remaining part of the exposure time. The dose value is collected for the whole exposure time. If you need to change the sensitivity, delay, or/and measuring window, tap to show the settings and make your choices.

In the case of dental panoramic system the situation is somewhat different. Here the kV and radiation waveform often is very well regulated.

The challenges instead arise for the mechanical setup needed to position the detector in right position. The small and narrow field is only a few millimetres. The Piranha detector has very narrow detector area and is very thin and a special holder (optional) can be used to position the Piranha without any problem. Panoramic units that use digital detectors have much smaller detector area and magnets cannot and should not be used close to the detector area.

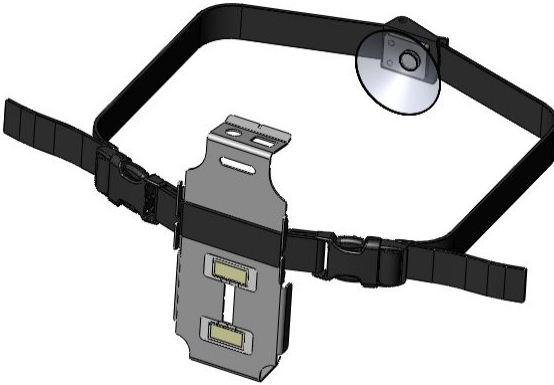

Special fixation rods for the head should be placed so that they are not in the beam during the scan or can hit the detector. In most cases the control panel have a special scan mode without X-ray so the mechanical set up can be tested.

Another important issue to be aware of is that a dental panoramic system normally compensates the for the thicker penetrating neck region in the patient when it makes its scan. This means that some units actually increase its tube voltage a short moment during the scan, other use different mA or scan speeds when the scanning beam passes the neck region. Newer digital system can actually measure patients X-ray beam attenuation dynamically and change the output level automatically during the scan.

The Piranha has addressed these challenges. Since the panoramic scan has an exposure time of about 10 to 20 seconds, the Piranha is set up to continuously update the display during the scan. It is preferable to select a single parameter display and angle the Palm holder, making it easy to read the values during the scan, from a distance.


You can always trust the kV reading of a dental measurement. This is true since Piranha can automatically detect whether the detector area is not fully uniformly radiated or not by means of the *Position Check*, and also compensates for the beam filtration during the scan.

You can either select the tube voltage as single parameter or together with dose, dose rate, and exposure time. As complementary information an estimation of the total filtration in the beam and type of waveform are made. This features use the kVp

filter R1[4] that also is the default kV range 55 - 105 kV when the instrument is turned on. The displayed dose value has very little energy dependence since it is automatically compensated for each exposure since the kV, estimated filtration and the waveform are measured.

4.6.1 Waveforms

The example below explains why a change of delay change the value of measured kVp and the kV and radiation waveform on a one phase dental unit.

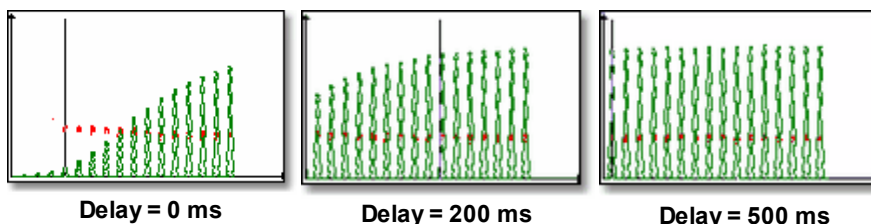

From the study of the above three screens several conclusions can be made:

1. A stable output level is not reached until after approximately 200 to 300 ms (20 to 30 pulses for a 50 Hz main based dental unit).
2. The exposure time is depending on the definition of the trig level.
3. The kVp value in the RTD is related to selected delay and window and is several kV higher in the beginning of the exposure at the same time as the radiation level is relative low here. Therefore a delay of 200 ms is default for the Piranha.

Note!

If the signal to the detector is too low to give a correct kV value, this part of the waveform will be blank. This is the reason why only the upper parts of the kV waveform is displayed. To see more of the waveform, change the kV range.

4.7 CT


4.7.1 CT kVp

To measure kVp on a CT is many times difficult since with most meters it is required to stop the tube in the top position at the same time as the table is not moving. This can normally not be obtained using an available standard clinical program. Instead a service mode must be used. Another problem is to "find" the beam, especially when using a small slice width. All these problems are minimized when using the Piranha since it can "move with the table" through the beam while the tube is in the top position. This is can easily be obtained by measuring while a topogram (scout/pilot image) is taken. A topogram is obtained with a moving table and a stationary tube, normally in the top position. The topogram is normally used to provide information for the actual CT scan. It is recommended to use a slice width of 3 mm or wider. That is, if selectable use as large slice width as possible.

You may also want to use the *Timed* mode to allow measurements on moving CT machines, see section [Update Modes](#) ^[43].

To measure CT kVp:

1. Set up the Piranha and the handheld computer according to the description in [Setting Up the Piranha](#) ^[10].

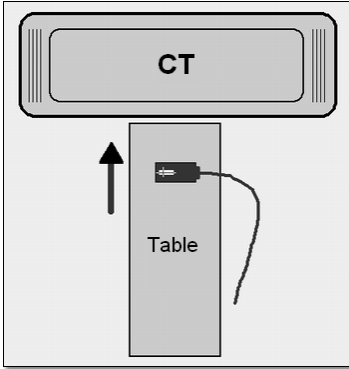

2. Place the detector on the patient bed in a region that is irradiated during the topogram process. Place the Piranha in the direction indicated by the figure below. That is, the Piranha detector surface rectangle should be placed perpendicular to the scanning direction.

You may use the lasers to align the Piranha correctly.

3. Select **Tube voltage**.

The following calibrations are available:

C1 = W/3.0 mm Al

C2 = W/3.0 mm Al + 0.25 mm Cu (optional)

C3 = W/3.0 mm Al + 1.2 mm Ti (optional)

C4 = GECT

C5 = Toshiba Aquillion

4. Make a check of the position of the Piranha by using the *Position Check* function.

5. Set up the CT to make a topogram.

6. Start the topogram program. If the procedure includes more than one topogram you may abort after the one taken with the tube in the top position.

If the Piranha is positioned in a correct way, the position is accepted and the real-time display is shown again. If not check the position of the Piranha and/or increase the slice width if possible.

7. You are now ready to measure. Repeat the topogram program to measure kVp.

The exposure time you measure is not related to the actual "radiation time". It is the time it takes for the detector to "pass through" the CT X-ray field when the table moves when it is acquiring the topogram.

4.8 Tube Current Probes

The mAs probes are used to measure mAs (current time product) and mA (tube current). Tube current is normally measured only for fluoroscopy or when long exposure times are possible to allow read-out during the exposure. When tube current is presented for exposures it has been calculated from the measured mAs and from measured exposure time.

For pulsed fluoroscopy it is possible to measure pulse mA in addition to the mA value. The difference between the pulse mA and the traditional mA is explained in the picture below.



(Note that for DC waveform, pulse mA and mA gives the same value.)

You can measure mAs as a single parameter or multi-parameter together with the Piranha. When using only the mAs-probe the measurement always starts when the mAs-probe detects a signal. When using multi-parameter you can choose to trig individually or to trig with the Piranha:

Individually: The mAs-probe starts to measure as soon as the tube current is detected. The Piranha starts to measure as soon as it detects the radiation. Normally will the mAs-probe start to measure first since tube current first charges the HV cables before it "reaches" the tube and radiation is generated.

Piranha: Both the mAs-probe and the Piranha starts to measure at the same time; when the Piranha detects radiation. This is the easiest way to measure since there is almost no risk for false triggering.

It is important to be aware of that measured mAs-values may differ depending on which trig method is used. Especially when measuring low mAs values the difference may be significant when comparing the two methods or comparing to "traditional" mAs meters. The value you get when triggering on the tube current (Individual trig) corresponds to the total mAs supplied from the generator. A part of that has been used to charge the cables and the rest has reached the tube and contributed to the exposure and the image. When you use Piranha trig you measure only the mAs that actually contributes to the exposure and the generation of the image.

The discussion above is generally true for the invasive MAS-1 probe since it is connected in the transformer and measures "all" current. The non-invasive probes, MAS-2 and MAS-3, can be placed anywhere on the HV cable. If they are placed close to the tube they will measure only the current that floats through the tube and

contributes to the radiation and the choice of trig source will have limited influence on the measured values.

When measuring on fluoroscopy the trig source has no influence and it is recommended to trig on the Piranha.

4.8.1 MAS-1, Invasive mAs Probe

This section describes how to measure the tube current and charge as a single parameter. This means that the measurement starts when the mAs-probe detects the tube current. The measurement is performed in the same way if you use a multi-parameter display. In that case the default trig is the Piranha and what is said below about false triggering can be ignored.



The MAS-1 probe provides an invasive way to measure mA and mAs on X-ray generators. The MAS-1 probe should be connected to the X-ray generator mAs socket. The figure to the left shows the Piranha MAS-1 Probe. Read the MAS-1 User's Manual for a detailed description on how to connect it.

To measure tube charge (mAs) with the Piranha MAS-1 Probe

1. Connect the MAS-1 probe to the X-ray generator as described in the MAS-1 User's Manual.
2. Connect the MAS-1 probe to the external probe input.
3. Place the Piranha internal detector in the X-ray field as for a normal exposure and continue measure as for a standard measurement.

If you get a message about **Negative Signal**, the current is floating in the wrong direction in the mAs probe. Switch the two connectors that are connected in the mAs measuring socket, do a **Reset** and make a new exposure.

Since the MAS-1 probe is connected in the X-ray generator false triggering may occur due to electrical noise when the pre-heat is started and the anode starts to rotate. If you get incorrect or inconsistent results try the following:

- First start anode rotation without firing the exposure.
- While the anode is rotating do a **Reset** or press the corresponding button. Make the exposure when the reset procedure is finished.

The MAS-1 probe can also be used for measurement of tube current during fluoroscopy.


To measure fluoroscopy tube current with the Piranha MAS-1 Probe

Do as for tube charge (mAs) as described above, but please note that for low mA values the mAs and the exposure time may not be measured.

4.8.2 MAS-2, Non-invasive mAs Probe

This section describes how to measure mAs, using the Piranha in the X-ray beam to trigger the measurement. This means that the measurement starts when the Piranha detects the radiation.

The MAS-2 probe uses a current clamp probe to measure mAs and mA non-invasively.

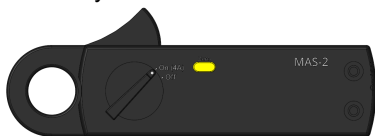

The MAS-2 probe is mostly used for mAs measurements since it is not sensitive enough to measure tube current on fluoroscopy. The figure to the left shows the MAS-2 probe without the cable.

The parameter mAs is available for most type of measurements but mAs is normally measured only for X-ray exposures.

To measure tube charge (mAs) with the Piranha MAS-2 Probe

1. Attach the MAS-2 probe to the high voltage cable close to the X-ray tube on the negative/cathode side. Please observe the polarity as indicated by an arrow inside the claws. Then power on the MAS-2 by turning the range switch to ON (4A) and make a Reset of the MAS-2 probe by pressing the yellow knob on the probe. Note that the "best" side for connection is to avoid the side that drives the tube rotation (it is usually driven on the anode side, creating current noise in that cable that the MAS-2 will pick up).
2. Connect the MAS-2 adapter cable to the Piranha.
3. Place the Piranha internal detector in the X-ray field as for a normal exposure and continue measure as for a standard measurement.
4. Make sure to use the Piranha internal detector as trig source. This will make the system trigger on the radiation, and thereby avoid self-triggering by ground loops in the high voltage cables.
5. Do a **Reset** before the first exposure.
6. Make an exposure.

If you get a message **Negative Signal**, the mAs probe is probably connected in the wrong direction. Change the direction of the mAs probe, press the Reset button on the mAs probe, do a **Reset**, and make a new exposure.


If using Individual trig

Since the MAS-2 probe is based on measurement of magnetic flux, false triggering may occur due to electrical noise when the pre-heat is started and the anode starts to rotate. If you get incorrect or inconsistent results try the following:

- First start anode rotation without firing the exposure.
- While the anode is rotating tap **Reset** or press corresponding button. Make the exposure when the reset procedure is finished.

4.9 Light Measurement

The Piranha Light Probe is a highly sensitive light detector. It has two different adapters to measure the quantities luminance and illuminance. The most common applications for the Piranha Light Probe are luminance (cd/m^2) measurements on CRTs (monitors) and viewing boxes, and illuminance (lx) measurements of ambient light in a room or in front of a CRT. Read the Piranha Light Probe User's Manual for a detailed description of practical use and explanation of the theory behind the units and quantities of light.

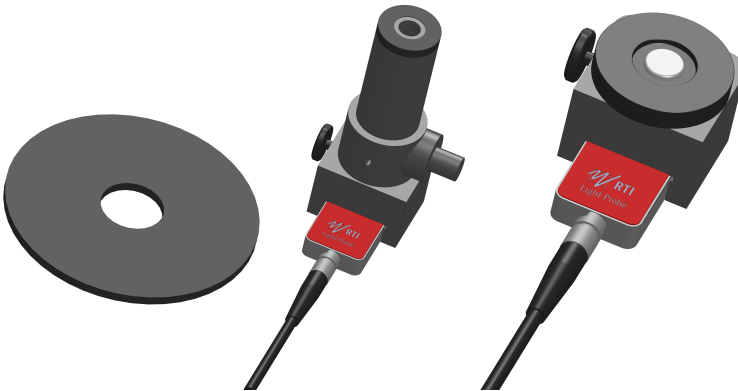

The monitor adapter is shown to the left and the lux adapter to the right.

Note! It is recommended to keep the monitor adapter's inside and lux adapter's white surface clean from dust to meet the specifications. Use a lint-free cloth or blow air to remove any dust. Please see the Light Probe manual (available on the web or Product CD/USB) for detailed information on cleaning.

4.9.1 Luminance - Monitor/Viewbox (cd/m²)

Read the Piranha Light Probe User's Manual to get information about how to do different type of measurements and how to use the different adapters.

To measure luminance (cd/m²):

1. Attach the monitor adapter to the Piranha Light Probe as described in the Piranha Light Probe User's Manual.
2. Connect the Piranha Light Probe to the Piranha.
3. Set up the Piranha and the handheld computer according to the description in [Setting Up the Piranha](#) ^[10].
4. Make a Reset. Place the Piranha Light Probe onto the surface where you want to measure the luminance.
5. Press and hold the shutter. Read the value. You can now move the Piranha Light Probe to other points and continue measuring the light.

4.9.2 Illuminance - Ambient Light (lx)

Read the Piranha Light Probe User's Manual to get information about how to do different type of measurements and how to use the different adapters.

To measure illuminance (lx):

1. Attach the lux adapter to the Piranha Light Probe as described in the Piranha Light Probe User's Manual.
2. Connect the Piranha Light Probe to the Piranha.
3. Set up the Piranha and the handheld computer according to the description in [Setting Up the Piranha](#) ^[10].
4. Cover the white light-sensitive area of the Piranha Light Probe to shield off all light (you may use the rubber part that comes with the Piranha Light Probe-M if available). It is very important that you shield off all light. Then make a **Reset**. After that you can remove the shield and place the light detector where you want to measure the ambient light.
5. Read the value. You can now move the Piranha Light Probe-L to other points and measure the ambient light.

Chapter 5

Problems and Solutions

5 Problems and Solutions

5.1 Troubleshooting

Before contacting your distributor or RTI, please check the following tips.

A. Check the RTI web page for updates <http://www.rti-group.com>.

B. Run through the checklist below.

The Piranha does not work

Check that the motor moves properly or can beep. (QABrowser: **Setup | System Test**).

The Piranha moving filter seem to have stuck

Try the following suggestions:

1. Make sure to attach the external power supply, that will give extra power to the motor.
2. If you have a QABrowser, start it and run the filter test. (QABrowser: **Setup | System Test**)
3. Hold the Piranha in your right hand by the cable edge.
4. When the motor is trying to move, tap the Piranha's left long edge in the palm of your left hand until it comes free.

The electrometer does not give a reading

Check:

1. That the correct input connector is used and connected.
2. That probe cables look healthy.
3. If using the Chamber Adapter and an ion chamber, check that the power and bias indicators on the Chamber Adapter are lit.

The electrometer gives numerous trig indications

If you get the trig indicator ("play" symbol) when there is no signal:

1. Press reset.
2. If it comes over and over you may need to increase the trig level, by raising the threshold, see topic [Measurement Settings](#) ^[35].

The electrometer or Piranha gives too low dose rate

If you get to low dose rate readings or too short irradiation times for short exposures:

1. Check that you are measuring with a good geometry, where the incoming radiation is perpendicular to the detector surface. See [Specifications, Piranha](#) ^[12] for details.

Installation or upgrade of RTI Updater failed

Make sure you are not having a restricted user account (on Windows XP, 2000, or newer). You will need to have access to an administrative account to install the software, see section [Windows Restricted User Accounts](#)^[74] for details.

My Piranha cycles its status indicator quickly between colours and does not measure

Your system is in Bootloader mode. Please run RTI Updater to correct this.

My Piranha beeps twice quickly when starting

The Piranha normally beeps once when starting. If it beeps twice quickly when powering on, run RTI Updater to correct this.

Bluetooth only shows "Unknown device" when trying to connect

Depending on Palm model it may take a little while for the serial number to appear, the Piranha will appear as "Unknown device". Normally it will show the serial number if you wait a while.

How do I change from Gray to Röntgen units?

You can set this for all tests (**pull-down menu | Setup | Units**), see Units Setup. It is also possible to set mixed units for a test and save as Favourites. All measurements settings, QABrowser settings, set values, and selected units will be saved with the Favourite.

C. Contact your local representative or see Notice for contact information to RTI Group AB. The more information you supply will help us to get a quick solution to your problem. Examples of useful information is screen dump pictures, exact error message texts, log files, etc. You may also use the auto-report function described in the section How To Report a Problem.

5.2 Bluetooth

Bluetooth is a wireless way of communication between your PC/handheld and your meter. The Piranha has built-in support for this. A Palm OS handheld also has built-in support for Bluetooth but a PC may or may not have built-in support for Bluetooth. If the PC does not have built-in Bluetooth support you will need a Bluetooth adapter (that you connect to the USB port) for instance the D-Link DBT-120 or DBT-122.

The range of Bluetooth is about 10 meters (32 feet) in free air for a class 2 Bluetooth adapter (like the D-Link DBT-122), for a class 1 Bluetooth adapter (like the Targus ACB20EU) the theoretical range is up to 100 metres. This can be significantly shorter if there are walls and other objects obstructing the signal.

Bluetooth and a Palm OS handheld works out of right out of the box, while using Bluetooth and PC may, depending on your PC, the BT drivers, and the Windows version, requires some work. If it is possible for you to use a USB cable with your PC and meter then that will often be more robust (but less convenient).

5.2.1 Bluetooth Passkey

There are two different ways to use Bluetooth with you meter, without a passkey and with a passkey (also called PIN code, authentication, and Bluetooth security code). All meters and accessories that are delivered from RTI from the first quarter of 2010 are configured to use a passkey (0000).

Drawbacks of using a Passkey

If you enable a passkey you might experience some drawbacks.

- If you use your meter (with Bluetooth communication) with more than one PC or with a PC and a Palm OS handheld you might need (depending on your hardware) to add the meter (also called to pair a device, or to add as a trusted device) with the PC/handheld every time you have used another PC or handheld with the meter.
- With a Palm OS handheld you cannot just simply start the QABrowser with Bluetooth. You need to first add the meter as a trusted device. And if you have used the meter with another handheld or PC (using Bluetooth) you will need to add the meter as a trusted device again.

New hardware which supports multiple devices

Piranha with product version 3.0 (which started shipping around spring 2009) and higher, and product version 2.5.4 supports multiple Bluetooth devices, which means that you can use it with both a PC and handheld without additional steps.

Advantages of using a Passkey

- Works with Windows Vista and Windows 7,8,10.
- Increased security

The main reason to change so that you use a passkey is if you are going to use Bluetooth communication and Windows 7-10, Vista, or Windows XP and a Bluetooth adapter where you cannot disable authentication. But we recommend that you use a USB cable if you can.

If you want to use a passkey and have previously not been using one, you need to reconfigure your Piranha. Detailed instructions are available later on in this manual.

5.2.2 Enable Bluetooth Passkey

Before you enable the use of a Bluetooth passkey please read the section about Bluetooth passkeys and the drawbacks of using a passkey.

You enable and disable passkeys by using the *RTI Updater* application (requires version 2008.6A or later) which is available on your RTI Software & Documentation Media and the RTI website (www.rti-group.com).

1. Connect your meter using the USB cable.
2. Launch RTI Updater from the "RTI Electronics" folder in the Windows start menu.

3. Select the USB connection. You cannot use a Bluetooth connection when you want to change the Bluetooth configuration. A Bluetooth connection is shown as **COMxx**.
4. Wait for RTI Updater to finish with the startup procedure. After a while when it is ready the **Start** button will be enabled.
5. Select **Settings | Advanced** from the menu.
6. Now select the **Tools** menu.
7. If you want to enable the use of a passkey select **Enable Bluetooth Passkey**. If you experience problems with this, then you probably need to update you meter firmware before you can enable the Bluetooth passkey. Please run the available updates (by pressing **Start**) and then restart the application again to enable the Bluetooth passkey.
8. Follow the onscreen instructions.

If you want to disable the Bluetooth passkey do the same thing but select **Disable Bluetooth Passkey** instead. If you are asked for a PIN code during the startup do **not** enter a PIN code, just cancel instead. Because if you do, you will have activated your Bluetooth and RTI Updater cannot reconfigure the Bluetooth module when it is active.

5.3 Windows Restricted User Accounts

If you only have a restricted user account (e.g. not an account with administrator privileges) for your computer you will need to contact your network administrator so that the administrator can help you with installing the software needed for your Piranha.

If you are an administrator trying to install the software for a user, read the instructions below in order to find out what you need to do in order to enable the restricted user to be able to run the RTI applications.

5.3.1 Which Applications are Needed?

The necessary applications differs depending on the user's needs.

Ocean (Windows platform)

Ocean use only requires that you install the Ocean software, but also note that there is another utility (RTI Updater) that the user may need in the future, in order to update the software located in the Piranha instrument (firmware).

RTI Updater

The RTI Updater application is used for updating the built-in software (firmware) of the Piranha. This is needed if the user is going to update either Ocean software in the future, since updates to that applications usually also requires that the firmware of the Piranha is updated.. Software updates for the different applications are available for download from the RTI website (www.rtigroup.com).

5.3.2 Installing the Software

Start by logging in as an administrator on the computer that you wish to install the software on.


If you have just purchased your Piranha and are a new user, you can use the Piranha Media that is shipped with your Piranha.


- If you are installing Ocean, goto **Install** and select **Ocean Installer** to start the installation. This will install the Support File Package, the Detector Manager, and RTI Updater.
- If you also need to install the QABrowser, then goto **Install** and select **Appropriate Version**.

For more information about the different installers please refer to the [Updating the Piranha Firmware](#)^[31] topic and the Installation chapter of the Ocean manual. Note that the programs will be available for all users of the computer. After the installations are done you will need to give these users some additional permissions to the directory where you installed the software, see [Setting Necessary Rights](#)^[75].


5.3.3 Setting Necessary Rights

The applications from RTI requires that you give the users "Read & Execute" and "Write" permissions to the directory where you installed the applications. Since all applications by default are installed in an "RTI Electronics" directory in the "Program Files" directory, the easiest way to accomplish this is by giving the user those permissions to the entire "RTI Electronics" directory. Do that by using Windows Explorer and navigate to the program files directory and right click the "RTI Electronics" folder and select "Properties".

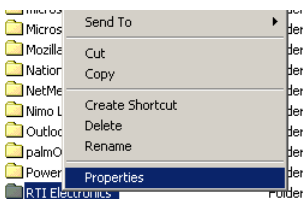

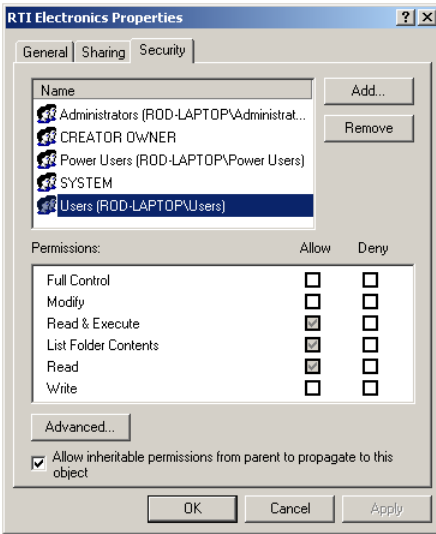

In the properties dialogue, go to the **Security** tab and make sure that the checkboxes for both "Read & Execute" and "Write" are checked for the user group or user that will be using the software.

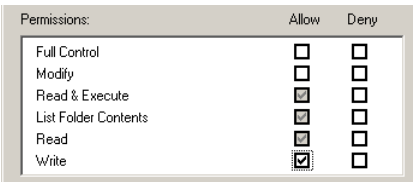

If they are not checked, check them and then click **OK**.

5.4 How To Report a Problem

There is a way of automatically sending technical support information to RTI when you are experiencing problems with the Ocean software.

This is described in the Ocean manual or help file under "Create support file".

Please try to include a good description of the problem. **Tip!** The more information you give, we will have better chances of reproducing the problem and finding a solution to it.

Chapter 6

Glossary

6 Glossary

Absorbed dose (D)

The energy imparted per unit mass by ionizing radiation to matter at a specified point. The SI unit of absorbed dose is joule per kilogram (J/kg). The special name for this unit is gray (Gy). The previously used special unit of absorbed dose was the rad. 1 rad = 0.01 Gy. 1 Gy = 100 rad. (See Report No. 82, NCRP, 1985b.)

SI unit: Gy = J/kg

Absorbed dose rate (D')

absorbed dose per unit time. Absorbed dose rate is determined as the quotient of dD by dt , where dD is the increment of absorbed dose in the time interval dt : $D' = dD/dt$. A unit of absorbed dose rate is any quotient of the gray or its multiples or submultiples by a suitable unit of time (Gy/s, mGy/h, etc.).

SI unit: Gy/s = J/kg·s

Absorption, energy

Phenomenon in which incident radiation transfers to the matter which it traverses some or all of its energy.

Activity

The number of nuclear transitions occurring in a given quantity of radioactive material per unit time. The SI unit of activity is s^{-1} . The special name for the unit of activity is becquerel (Bq). The previously used special unit of activity was the curie (Ci). 1 Bq = 2.7×10^{10} Ci. 1 Ci = 3.7×10^{10} Bq. (See Report No. 82, NCRP, 1985b.)

SI unit: Bq = s^{-1}

Additional filtration

ADDED FILTERS and other removable materials in the RADIATION BEAM which are between the RADIATION SOURCE and the PATIENT or a specified plane.

See also filter.

Air kerma

See kerma.

Aluminium equivalent or Aluminium Attenuation Equivalent (AAE)

The thickness of aluminum affording the same attenuation, under specified conditions, as the material in question.

Anode

In a X-ray tube, electrode to which electrons forming a beam are accelerated and which usually contains the target.

Aperture

(e.g., for computed tomography) - the opening in the collimation that allows radiation to reach the detector.

Area exposure product

Product of the area of a cross-section of a radiation beam and the averaged exposure over that cross-section.

SI unit: Gy·m²

Attenuation

The reduction of radiation intensity upon passage of radiation through matter.

Automatic exposure control (AEC)

In an X-ray generator, mode of operation in which one or more loading factors are controlled automatically in order to obtain at a preselected location a desired quantity of radiation.

Automatic exposure rate control

In an X-ray generator, mode of operation in which the rate of emitted radiation is controlled automatically by control of one or more loading factors in order to obtain at a preselected location and in a preselected loading time a desired quantity of radiation.

Beam limiting device

Device to limit the radiation field.

Becquerel (Bq)

The special name for the SI unit of activity. One becquerel is one reciprocal second or 1 s⁻¹. 3.7×10^{10} Bq = 1 Ci.

Bootloader

General: a program that does the job of loading the OS kernel of a computer.

Piranha bootloader: Miniature program stored in cabinet and modules which normally just starts the Firmware. It is used more when the Firmware is updated. See Firmware.

Centigray

0.01 gray. 1 cGy equals one rad.

Cinefluorography

The production of motion picture photographic records of the image formed on the output phosphor of an image intensifier by the action of X-rays transmitted through the patient (often called cineradiography).

Cineradiography

Indirect radiography of moving objects usually in rapid series on cine film.

Collimator

See beam limiting device.

Compensating filter

Filter used in order to modify the distribution of absorbed dose rate over the radiation field.

Computed tomography (CT)

An imaging procedure that uses multiple X-ray transmission measurements and a computer program to generate tomographic images of the patient.

Continuous mode

For an X-ray generator, mode of loading an X-ray tube continuously as in radiotherapy or in radioscopy.

Conversion factor (of an image intensifier)

The quotient of the luminance of the output phosphor of the image intensifier divided by the kerma rate at the input phosphor.

SI unit: $\text{cd}/\text{m}^2 / \text{Gy}/\text{s} = \text{cd}\cdot\text{s}/\text{Gy}\cdot\text{m}^2$

CT

See Computed Tomography

CT number

One of a set of numbers on a linear scale which are related to the linear attenuation coefficients calculated by a computed tomographic device. One of the specific set of CT numbers on a scale from -1000 for air to +1000 for bone, with water equal to zero, which is called a Hounsfield unit.

Curie (Ci)

The previously used special unit of activity equal to 3.7×10^{10} per second. $1 \text{ Ci} = 3.7 \times 10^{10} \text{ Bq}$.

Dead man switch

A switch so constructed that a circuit-closing contact can be maintained only by continuous pressure on the switch.

Dental panoramic radiographic

Direct radiography of a part of or the complete dentition by the use of an intra-oral X-ray tube. See also Orthopantomography.

Diagnostic source assembly

A diagnostic source housing (X-ray tube housing) assembly with a beam limiting device attached. This assembly shall be so constructed that the leakage radiation air kerma measured at a distance of one meter from the source does not exceed 1 mGy (0.1 rad) in one hour when the source is operated at its leakage technique factors. (See definition).

Digital radiography

A diagnostic procedure using an appropriate radiation source and an imaging system which collects processes, stores, recalls, and presents image information in a digital rather than analogue fashion.

Digital subtraction

An image processing procedure used to improve image contrast by subtracting one digitized image from another.

Dose equivalent (H)

A quantity, defined for radiation protection purposes, which is the product of the absorbed dose to the tissue and a quality factor "Q" determined by the properties of the radiation that produced the absorbed dose. For X-rays, gamma rays, and electrons, $Q = 1$ and dose equivalent values are numerically equal to absorbed dose values when consistent units are used for both quantities. The SI unit for dose equivalent is joule per kilogram. The special name for the SI unit of dose equivalent is sievert (Sv). The previous special unit of dose equivalent was the rem. One sievert equals 100 rem.

SI unit: Sv = J/kg

Dose rate meter

Radiation meter intended to measure absorbed dose per unit time.

Dosimeter

Radiation meter intended to measure absorbed dose.

Effective dose equivalent (HE)

Quantity used to express the weighted DOSE EQUIVALENT to the whole body when it is irradiated non uniformly or partially.

Exposure (X)

A measure of the quantity of X-ray or gamma radiation based upon its ability to ionize air through which it passes. The SI unit of exposure is coulomb per kilogram. The previously used special unit of exposure was röntgen (R). $1 \text{ R} = 2.58 \times 10^{-4} \text{ C} \cdot \text{kg}^{-1}$ (exactly). The physical quantity exposure is now replaced by the quantity kerma in air. An exposure of 114.1 R is equal to an Air Kerma of 1 Gy. That means that the value in R should be multiplied by 8.76 to get the Air Kerma in mGy.

SI unit: C/kg

Exposure rate (X')

Exposure per unit time. Exposure rate is determined as the quotient of dX by dt , where dX is the increment of exposure in the time interval dt : $X' = dX/dt$. A unit of exposure rate is any quotient of the unit of exposure or its multiples or submultiples by a suitable unit of time ((C/kg)/s, (mC/kg)/h, etc.).

SI unit: C/kg · s

Filter

In radiological equipment, material or device provided to effect filtration of the radiation beam.

SI unit: mm

Filter: Inherent filter

The filter permanently in the useful beam; it includes the window of the X-ray tube and any permanent enclosure for the tube or source. Replaced by term Permanent filter

Filter: Added filter

Filter in addition to the inherent filtration.

Filter: Permanent filter

The filter permanently in the useful beam; it includes the window of the X-ray tube and any permanent enclosure for the tube or source.

Filter: Total filter

The sum of the permanent and added filters.

Firmware

General: The operating system and software installed on a small device. Sometimes called embedded software.

Piranha firmware: Program stored in cabinet and modules which handles all control of measurement electronics. Can be updated, then a special part of the firmware called bootloader, is used. See Bootloader.

Fluorography

The production of a photographic record of the image formed on the output phosphor of an image intensifier by the action of X-rays transmitted through the patient.

Fluoroscopy

Technique of radioscopy by means of a fluorescent screen.

Focal spot, effective

The apparent size of the radiation source region in a source assembly when viewed from the central axis of the useful radiation beam.

SI unit: dimensionless (corresponding to a dimension in mm)

Framing

In cinefluorography, the registration of the circular image of the output phosphor on the rectangular film element or frame.

Gantry

The moveable patient table used for CT.

Geometric unsharpness

Unsharpness of the recorded image due to the combined optical effect of finite size of the radiation source and geometric separation of the anatomic area of interest from the image receptor and the collimator.

Gray (Gy)

The special name for the SI unit of absorbed dose, kerma, and specific energy imparted equal to one joule per kilogram. One gray equals one joule per kilogram. The previous unit of absorbed dose, rad, has been replaced by the gray. One gray equals 100 rad.

Half-value layer (HVL)

Thickness of a specified substance which, when introduced into the path of a given beam of radiation, reduces the kerma rate by one-half.

SI unit: mm

Heel effect

Non-uniform intensity observed because a small fraction of the X-ray beam emitted in a direction nearly parallel to the angled target surface must pass through more target material before escaping from the target than does the major portion of the beam which is emitted more perpendicularly. (Note: In addition to the non-uniform intensity the angled target also produces non-uniform image resolution due to variations in apparent focal spot size as viewed from various positions on the film).

Hounsfield units

See CT number.

Image intensifier

An X-ray image receptor which increases the brightness of a fluoroscopic image by electronic amplification and image minification.

Image receptor

A system for deriving a diagnostically usable image from the X-rays transmitted by the patient. Examples: screen film system; stimuable phosphor; solid state detector.

Inherent filtration

Filter between the radiation source and the output window of the X-ray equipment. See filter.

Initial X-ray tube voltage

In a capacitor discharge X-ray generator, X-ray tube voltage at the beginning of the loading of the X-ray tube.

Installation

A radiation source with associated equipment, and the space in which it is located.

Interlock

A device used to assure proper and safe use of a radiation installation by monitoring (usually by electrical devices) the status, presence or position of various associated devices such as source position, collimator opening, beam direction, door closure, filter presence, and preventing the production or emission of radiation if the potential for an unsafe condition is detected.

Ionization

Formation of ions by the division of molecules or by the addition or removal of electrons from atoms or molecules.

SI unit: C, Coloumb

Ionization chamber

Ionization detector consisting of a chamber filled with a suitable gas, in which an electric field, insufficient to induce gas multiplication, is provide for the collection at the electrodes of charges associated with ions and the electrons produced in the sensitive volume of the detector by ionizing radiation.

Ionization constant

For air the ionization constant $W/e = 33,97 \text{ J/C}$. The ionization constant is used to get the correspondence between exposure and air kerma. See Roentgen and Gray for more information.

Ionization detector

Radiation detector based on the use of ionization in the sensitive volume of the detector.

Irradiation time

Irradiation time is usually the time a rate of a RADIATION QUANTITY exceeds a specified level. Irradiation time is sometimes called Exposure time.

SI unit: s, second

Kerma (K)

The sum of the initial kinetic energies of all the charged ionizing particles liberated by uncharged ionizing particles per unit mass of a specified material. Kerma is measured in the same unit as absorbed dose. The SI unit of kerma is joule per kilogram and its special name is gray (Gy). Kerma can be quoted for any specified material at a point in free space or in an absorbing medium. Typically the kerma is specified in air.

SI unit: $\text{Gy} = \text{J/kg}$

Kerma rate (K')

Kerma per unit time. Kerma rate is determined as the quotient of dK by dt , where dk is the increment of kerma in the time interval dt : $K' = dK/dt$. A unit of kerma rate is any quotient of the Gray or its multiples or submultiples by a suitable unit of time (Gy/s, mGy/h, etc.).

SI unit: $\text{Gy/s} = \text{J/kg} \cdot \text{s}$

Kilovolt (kV)

A unit of electrical potential difference equal to 1000 volts.

kVp

See Peak tube voltage

Lead equivalent

The thickness of lead affording the same attenuation, under specified conditions, as the material in question.

Leakage radiation

All radiation coming from within the source assembly except for the useful beam.

(Note: Leakage radiation includes the portion of the radiation coming directly from the source and not absorbed by the source assembly, as well as the scattered radiation produced within the source assembly).

Magnification imaging

An imaging procedure carried out with magnification usually produced by purposeful introduction of distance between the subject and the image receptor.

Measured value

Estimate of the true value of a quantity, derived from the indicated value of a meter after applying all relevant correction factors.

Medical diagnostic radiology

Medical diagnosis using ionizing radiation.

Modulation transfer function (MTF)

A mathematical entity that expresses the relative response of an imaging system or system component to sinusoidal inputs as a function of varying spatial frequency, which is often expressed in linepairs per millimetre (lp/mm), the correct unit is however m^{-1} (or often mm^{-1}). The reference value most commonly used is that for zero frequency. The MTF can be thought of as a measure of spatial resolution of the detector system.

SI unit: m^{-1}

Monitor, personnel

See personnel monitor.

Occupancy factor (T)

The factor by which the workload should be multiplied to correct for the degree of occupancy (by any one person) of the area in question while the source is in the "ON" condition and emitting radiation. This multiplication is carried out for radiation protection purposes to determine compliance with the dose equivalent limits.

Operator

Any individual who personally utilizes or manipulates a source of radiation.

Orthopantomography

Orthopantomography (also called OPG or Panorama) is a radiographic procedure that produces a single image of facial structures including the upper and lower dentition jaws and their supporting structures and bones. Mostly used in dental applications. An OPG ("orthopantomogram") gives a panoramic view of the mouth, giving information on the teeth and the bones of the upper and lower jaw.

Particle fluence

Number of particles incident on a sphere, divided by the cross-sectional area of the sphere.

SI unit: m^{-2}

Personnel monitor

Also known as personal monitor. An appropriately sensitive device used to estimate the absorbed dose received by an individual.

Peak tube voltage \dot{U}_0 (kVp)

The peak value of the tube voltage (corresponding to the highest available radiation energy).

Phantom

In medical radiology, object behaving in essentially the same manner as tissue, with respect to absorption or scattering of the ionizing radiation in question. Phantom are used, for example, for simulating practical conditions of measurement:

- for purposes of radiation protection,
- for evaluating the performances to the diagnostic systems with respect to the radiation or to the object,
- for dosimetry.

Pixel

A two-dimensional picture element in the presented image.

Practical Peak Voltage (PPV)

The PPV is the constant potential producing the same image contrast as the waveform under test. PPV is defined in the IEC 61676 standard as: "The PRACTICAL PEAK VOLTAGE is based on the concept that the radiation generated by a high voltage of any waveform produces the same AIR KERMA contrast behind a specified PHANTOM as a radiation generated by an equivalent constant potential. The constant potential producing the same contrast as the waveform under test is defined as PRACTICAL PEAK VOLTAGE".

Primary protective barrier

See protective barrier

Protective apron

An apron made of radiation absorbing materials, used to reduce radiation exposure.

Protective barrier

A barrier of radiation absorbing material(s) used to reduce radiation exposure.

Protective glove

A glove made of radiation absorbing materials used to reduce radiation exposure.

Rad

The previously used special unit of absorbed dose. It is equal to 100 ergs per gram. 1 rad = 0.01 Gy (10^{-2} gray).

Radiation (ionizing)

Any electromagnetic or particulate radiation capable of producing ions, directly or indirectly, by interaction with matter. Examples are X-ray photons, charged atomic particles and other ions, and neutrons.

Ripple factor

The variation in the high-voltage expressed as the percentage of the maximum high-voltage across the X-ray tube during X-ray production: Ripple factor (%) = $100 \times (V_{\max} - V_{\min}) / V_{\max}$

Radiation protection survey

An evaluation of the radiation safety in and around an installation, that includes radiation measurements, inspections, evaluations, and recommendations.

Radiation receptor

Any device that absorbs a portion of the incident radiation energy and converts this portion into another form of energy which can be more easily used to produce desired results (e.g., production of an image). See image receptor.

Radiation source

The region and/or material from which the radiation emanates.

Radiogram

A film or other record produced by the action of X-rays on a sensitized surface.

Radiography

The production of images on film/image detector by the action of X-rays transmitted through the patient.

Receptor

See radiation receptor.

Receptor assembly

A radiation receptor in the specialized container necessary for the proper operation of the receptor.

Rem

The previously used special unit of dose equivalent. One rem equals 10^{-2} sievert (Sv).

Resolution

In the context of an image system, the output of which is finally viewed by the eye, it refers to the smallest size or highest spatial frequency of an object of given contrast that is just perceptible. The intrinsic resolution, or resolving power, of an imaging system is measured in mm^{-1} or line pairs per millimeter (lp/mm), ordinarily using a resolving power target. The resolution actually achieved when imaging lower contrast objects is normally much less, and depends upon many variables such as subject contrast levels and noise of the overall imaging system.

Roentgen (R) (or Röntgen)

The previously used special unit of exposure. $1 \text{ R} = 2.58 \times 10^{-4} \text{ C/kg}$. – Originally (Stockholm 1928) defined as "international R." (Symbol: r) and later (Chicago 1937) modified to: Roentgen- or γ -ray, that gives a charge of 1 esE from secondary emission in 0,001293 g of air.

This means that an exposure of one Roentgen will produce $2,58 \times 10^{-4}$ coulomb of ions of either sign per kilogram in air. Here the previously used physical quantity exposure has been replaced by kerma in air. See kerma. One R does not equal 1 cGy as the units C/kg and J/kg are different. To do this conversion the ionization constant for air must be used, which is 33,97 J/C. This is how it calculated: $1 \text{ Gy} = 1 \text{ J/kg} \hat{=} 1 \text{ J/kg} / (2,58 \times 10^{-4} \text{ C/kgR} \times 33,97 \text{ J/C}) = 114,1 \text{ R}$. An exposure of 114,1 R thus equals an Air Kerma of 1 Gy. That also means that the value in R should be multiplied by 8,76 to get the Air Kerma in mGy. (See also Exposure.)

Scattered radiation

Radiation that, during passage through matter is changed in direction. (It is usually accompanied by a decrease in energy.)

Serial radiography

A radiographic procedure in which a sequence of radiographs is made rapidly by using an automatic cassette changer, image intensifier/TV chain, etc.

Shutter

In beam therapy equipment, a device, attached to the X-ray or gamma-ray source housing to control the "ON" or "OFF" condition of the useful beam.

Sievert (Sv)

The special name for the SI unit of dose equivalent. One sievert equals one joule per kilogram. The previously used unit was the rem. One sievert is equal to 100 rem.

Signal-to-noise ratio

For video cameras, the ratio of input signal to background interference. The greater the ratio, the clearer the image.

Simulator

Diagnostic energy X-ray equipment used to simulate a therapy treatment plan outside the treatment room.

Slice

The single body section imaged in a tomography procedure.

Source

See radiation source.

Source-detector distance (SDD)

The distance measured along the central ray from the centre of the front surface of the source (X-ray focal spot or sealed radioactive source) to the active surface of the detector.

Source-to-image-distance (SID)

The distance measured along the central ray from the centre of the front of the surface of the source (X-ray focal spot of sealed radioactive source) to the surface of the image detector.

Source-surface distance (source-skin distance) (SSD)

The distance measured along the central ray from the centre of the front surface of the source (X-ray focal spot or sealed radioactive source) to the surface of the irradiated object or patient.

Spot film

A radiograph taken during a fluoroscopic examination for the purpose of providing a permanent record of an area of interest or to verify the filling of a void with contrast media.

Stray radiation

The sum of leakage and scattered radiation.

Survey

See radiation protection survey.

Target

The part of an X-ray tube anode assembly impacted by the electron beam to produce the useful X-ray beam.

Tenth value layer (TVL)

Thickness of a specified substance which, when introduced into the path of a given beam of radiation, reduces the kerma rate to one-tenth of its original rate.

Tomography

A special technique to show in detail images of structures lying in a predetermined plane of tissue, while blurring or eliminating detail in images of structures in other planes.

Topogram

For CT, prior to making the cross-sectional scans, the CT scanner is normally used to obtain one or more radiograph-like reference images, as a way of identifying and documenting where the scans are to be made. These so-called topograms are prepared by keeping the X-ray source and the detectors stationary, and dragging the specimen through the fan-beam by moving the table. Also called scout scans, pilot scans, or scanograms.

Total filtration

The total of inherent filtration and additional filtration.

Useful beam

The radiation which passes through the opening in the beam limiting device and which is used for imaging or treatment.

User

Physicians and other responsible for the radiation exposure of patients.

Voxel

A volume element in the object being imaged. The mean attenuation coefficient of the voxel determines the CT (Hounsfield) number of the pixel.

Whole body dose equivalent (Hwb)

The dose equivalent associated with the uniform irradiation of the whole body.

Workload (W)

The degree of use of a radiation source. For X-ray machines operating at tube potentials below 500 kV, the workload is usually expressed in milliamperere minutes per week. For gammabeam therapy sources and for photon-emitting equipment operation at 500 kV or above, the workload is usually stated in terms of the weekly

kerma of the useful beam at one meter from the source and is expressed in grays per week at one meter.

Xeroradiography

The production of an image on a xerographic plate (e.g., electrically charged selenium) by the action of X-rays transmitted through the patient. (xeromammography: Mammography carried out by the xeroradiographic process.)

X-ray tube

Evacuated vessel for the production of x-radiation by the bombardment of a target, usually contained in an anode, with electrons accelerated from a cathode by an electric field. Thus: Rotating anode X-ray tube. Double focus X-ray tube.

Index

Note!

Page references in this Index points to the first page of the section it is mentioned, not the exact page.

- 2 -

2002/96/EC 29

- A -

About 4
 Absorbed dose 78
 Absorption 78
 Active display messages 45
 Active Messages 45
 Activity 78
 Additional filtration 78
 Additional filtration (mammo) 38
 Administrative user 74
 Advantages of using a Passkey 73
 AEC 78
 After exposure 43
 After exposure update mode 38
 AGD 56, 60
 Air kerma 78
 Air kerma (Dose) 14, 15, 18
 Air kerma rate (Dose rate) 14, 15, 18
 Aluminium equivalent 78
 Ambient light 69
 AMX-4 36
 Analogue Out 50
 Anode 78
 Anode/Filter combination 54
 Mo/Mo 54
 Mo/Rh 54
 Rh/Rh 54
 W/Rh 54
 Aperture 78
 Atmospheric pressure 37
 Authentication, Bluetooth 73
 Auto reset 43
 Automatic exposure control 78
 Auto-power off 71
 Autoscaling 71
 Average Glandular Dose 56, 60

- B -

Battery
 charging times 12
 level 8, 12

 running time 8, 12
 status 8
 Battery charging indicator 8
 Beam Correction Factor 38, 59
 Beam limiting device 79
 Beam quality 50, 54
 Becquerel 79
 Blank screen 71
 Bluetooth 71, 72
 Indicator 7
 Passkey 73
 Passkey advantages 73
 Passkey disadvantages 73
 PIN code advantages 73
 PIN code disadvantages 73
 Security code 73
 Bootloader 31, 79

- C -

Calibrations
 View 34
 Camera thread 7
 CAS-6 10
 CAS-7 10
 cd/m² 69
 CE Declaration 30
 CE Mark 30
 Centigray 79
 Charging Times 12
 Chest w all distance 54
 Cine 51, 52
 HVL 52
 Quick-HVL 52
 Total filtration 52
 Cinefluorography 79
 Cineradiography 79
 Collimator 79
 Communication
 Bluetooth 12
 USB 12
 Compensating filter 79
 Compensation 50
 Compliances 29
 Compression paddle 38
 Equivalent thickness 36, 58
 Computed tomography 63
 Computed tomography (CT) 79
 Conditions 36, 50
 TP-factor 37
 Conformity Declaration 30
 Connector
 External probe 7
 Palm charging 7
 USB 7

Connectors 7
 Continuous 43
 Continuous mode 79
 Continuous update mode 38
 Conversion factor (of an image intensifier) 79
 CT 63, 79
 kVp 63
 CT Dose Profiler 28
 CT number 79
 CT specifications 18
 CT topogram 63
 CT-DP 28
 Curie (Ci) 79

- D -

Dead man switch 80
 Declaration of Conformity 30
 Default Unit 71
 Delay 38, 40, 41, 47, 63
 Start after 38
 Waveform 38
 Delay time 46
 Dental 61
 Dental panoramic radiographic 80
 Dental specifications 14
 Dental waveforms 63
 Detector area 12
 Detector Manager 34
 Detector settings 41
 Detector surface 7
 Detectors
 Managing 34
 Viewing 34
 Diagnostic source assembly 80
 Digital radiography 80
 Digital subtraction 80
 Direct radiography of a part of or the complete dentition by the use of an intra-oral X-ray tube. See also Orthopantomography. 80
 Disable Bluetooth passkey 73
 Display messages 45
 Active 45
 Passive 46
 Dose 51
 Mammography 56
 Dose equivalent (H) 80
 Dose Probe 7
 Dose Probe Specifications 26
 Dose rate meter 80
 Dose Sensitivity 40
 Dose/Pulse 19, 51
 Dosimeter 80

Draw backs of using a Passkey 73

- E -

Effective dose equivalent (HE) 81
 Enable Bluetooth passkey 73
 Entrance Skin Exposure 56, 60
 Entrance Surface Air Kerma 56, 60
 Equivalent thickness
 Compression paddle 36, 58
 Error 71
 Error messages 45
 ESAK 56, 60
 ESE 56, 60
 Estimated total filtration 14, 15, 18
 EU Directive 29, 30
 Exp. < Delay 46
 Exposure (X) 81
 Exposure rate (X') 81
 External probe connector 7

- F -

FCC 31
 Filter 81
 Filter: Added filter 81
 Filter: Inherent filter 81
 Filter: Permanent filter 81
 Filter: Total filter 81
 Filtration Additional(mammo) 38
 Firmware 4, 31, 81
 Fluorography 81
 Fluoroscopy 52, 81
 Fluoroscopy specifications 14
 Focal spot, effective 81
 Framing 81
 Free run 43, 44
 Free run update mode 38
 Using 44

- G -

Gantry 82
 Geometric unsharpness 82
 Good geometry
 HVL 58
 Gray (Gy) 82

- H -

Half-value layer (HVL) 82
 Handheld Reset 71
 Heel effect 10, 82
 Help 50
 High kVp 45, 46
 High signal 45, 46

Holder
 Piranha 10
 HotSync 71
 Hounsfield units 82
 How To
 Report a Problem 76
 HVL
 Cine 52
 Good geometry 58
 Stand 10

- I -

IEC 61267 20
 II dose rate 53
 Illuminance 69
 Image intensifier 53, 83
 Image receptor 83
 Indicator
 Battery charging 7
 Bluetooth 7
 Status 7
 Inherent filtration 83
 Initial X-ray tube voltage 83
 Input dose rate 53
 Installation 83
 Interlock 83
 Internal detector settings 35, 40
 Introduction 4
 Ionization 83
 Ionization chamber 83
 Ionization constant 83
 Ionization detector 83
 Irradiation time 14, 15, 18, 47, 83
 ISO 4037 20

- K -

Kerma 84
 Kerma rate 84
 Kilovolt (kV) 84
 kV Sensitivity 40
 kVp 14, 15, 18, 84

- L -

Lead equivalent 84
 Leakage radiation 84
 Light measurement 68
 Light probe 69
 Light Probe Specification 28
 Low kVp 45, 46
 Low Signal 45, 46
 Luminance 69
 lux 69
 lx 69

- M -

Magnification imaging 84
 Maintenance 31
 Mammography 54
 Dose 56
 Positioning Piranha 54
 Quick-HVL 57
 Mammography specifications 15
 Manufacturer's Declaration of Conformity
 30
 mAs 66, 67
 MAS-1 Probe Specifications 27
 MAS-2 Current Probe 67
 MAS-2 Probe Specifications 28
 Mean Glandular Dose 56, 60
 Measured value 84
 Measurement 51, 53
 Cine 51
 Dose 51
 Dose per pulse 51, 53
 Fluoroscopy 52
 Image intensifier 53
 Mammography 56
 Number of pulses 51
 Pulse rate 53
 Pulsed fluoroscopy 53
 Pulsed radiography 51
 Radiography 50
 Measurement Modes
 Overview 43
 Measurement settings 35
 Measuring principle
 Piranha 48
 Medical diagnostic radiology 84
 MGD 56, 60
 Min. output peak dose rate 19
 Minimum pulse width 19
 Minimum ripple 19
 Mo/Mo Anode/Filter combination 54
 Mo/Rh Anode/Filter combination 54
 Modes
 Of Measurement 43
 Modulation transfer function (MTF) 84
 Monitor 69
 Monitor, personnel 84
 Moving average 44

- N -

Necessary Rights 75
 Negative Signal 46
 Normalization distance 36, 58

- O -

Occupancy factor (T) 85
 Operating air pressure 12
 Operating temperature 12
 Operator 85
 OPG 61
 Orthopantomography 85
 Oscilloscope 50
 Overview of Measurement Modes 43

- P -

Panoramic dental 61
 Particle fluence 85
 Passive display messages 46
 Passkey advantages 73
 Passkey draw backs, Bluetooth 73
 PC requirements 5
 Peak tube voltage \dot{U}_o (kVp) 85
 Personnel monitor 85
 Phantom 85
 Physical dimensions
 Piranha 13
 PIN code advantages 73
 PIN code draw backs, Bluetooth 73
 PIN code, Bluetooth 73
 Piranha 10, 48
 Cable 10
 Holder 10
 Mammography positioning 54
 Physical dimensions 13
 Specifications 12
 Piranha internal detector 48
 Piranha Light Probe 68, 69
 Piranha MAS-1 Probe 66
 Piranha settings 35, 38
 Pixel 85
 Position check 51, 56, 61
 Post delay 38, 47
 Power switch 7
 Practical Peak Voltage (PPV) 85
 Prefixes, Unit 71
 Pre-pulse mammography 60
 Pressure
 Atmospheric 37
 Primary protective barrier 85
 Probes
 Managing 34
 Viewing 34
 Problem Report 71, 76
 Protective apron 85
 Protective barrier 85
 Protective glove 85

Pulse Measurements 51
 Pulse Rate 19, 37, 51
 Pulsed fluoroscopy 52, 53
 Pulsed radiography 51

- Q -

Quick-HVL
 Cine 52
 Mammography 57
 Radiography 51

- R -

Rad 86
 Radiation (ionizing) 86
 Radiation protection survey 86
 Radiation quality 20
 Radiation receptor 86
 Radiation source 86
 Radiation time 47
 Radiogram 86
 Radiography 50, 51, 86
 Quick-HVL 51
 Total filtration 51
 Radiography specifications 14
 Receptor 86
 Receptor assembly 86
 Recording time 43
 Reference conditions 12
 Rem 86
 Report a Problem 76
 Reposition Detector 46
 Requirements, PC 5
 Reset
 Bluetooth 71
 Handheld 71
 Reset time 43
 Resolution 86
 Restricted user 74
 Rh/Rh Anode/Filter combination 54
 Ripple factor 86
 Roentgen (R) (or Röntgen) 86
 Röntgen Unit 71
 RTI Detector Manager 34
 RTI Updater 31
 Running time 8, 12

- S -

Safety strap 7
 Scanning Beam Mammography 61
 Scatter factor 36, 58
 Scattered radiation 88
 SDD
 Normalization 36, 58

- Send Support Information 76
 - Sensitivity 40, 41
 - Dose/TF 40
 - kV 40
 - Serial radiography 88
 - Settings 35, 38, 40, 41, 50
 - Beam correction factor 38
 - Compression paddle 38
 - Conditions 35, 36
 - Delay 40, 41
 - Detector 41
 - Internal detector 35, 40
 - Piranha 35, 38
 - Post delay 38
 - Pulse rate 37
 - Sensitivity 40, 41
 - Threshold 41
 - Total Filtration 36
 - Trig level (time) 38
 - Trig source 38
 - Update mode 38
 - Waveform recording time 38
 - Waveform type 36
 - Window 40, 41
 - Shutter 88
 - Sievert (Sv) 88
 - Signal Extension Module 50
 - Signal-to-noise ratio 88
 - Simulator 88
 - Size 13
 - Slice 88
 - Source 88
 - Source-detector distance (SDD) 88
 - Source-surface distance (source-skin distance) (SSD) 88
 - Source-to-image-distance (SID) 88
 - Specifications
 - Air kerma (Dose) 14, 15, 18
 - Air kerma rate (Dose rate) 14, 15, 18
 - Battery Charging 12
 - Bluetooth 12
 - Communication 12
 - CT 18
 - Dental 14
 - Dose Probe 26
 - Dose/pulse 19
 - Estimated total filtration 14, 15
 - Fluoroscopy 14
 - Irradiation time 14, 15, 18
 - kVp 14, 15, 18
 - Light Probe 28
 - Mammography 15
 - MAS-1 Probe 27
 - MAS-2 Probe 28
 - Min. output peak dose rate 19
 - Minimum pulse width 19
 - Minimum ripple 19
 - Piranha 12
 - Power Source 12
 - Pulse rate 19
 - Radiography 14
 - Size 13
 - USB 12
 - Waveform recording time 19, 27
 - Weight 13
 - Spot film 88
 - Stand 10
 - Standards and Compliances 29
 - Start 31
 - Start after delay 38
 - Status indicator 7
 - Storage temperature 12
 - Stray radiation 88
 - Support 71
 - Support Information 76
 - Survey 88
- T -**
- Target 89
 - Temperature 37
 - Tenth value layer (TVL) 89
 - TF Sensitivity 40
 - Theory
 - Current reading 47
 - Delay 47
 - Dose rate reading 47
 - Irradiation time 47
 - Post delay 47
 - Radiation time 47
 - Waveform 47
 - Window 47
 - Threshold 41
 - Timed 43, 44
 - Timed update mode 38
 - Using 44
 - Tomography 89
 - Topogram 63, 89
 - Total filtration 36, 50, 89
 - Cine 52
 - Radiography 51
 - TP-factor 37
 - Trig level (time) 38
 - Trig source 38
 - Trigger 47
 - Troubleshooting 71
 - Tube current
 - Invasive 66
 - Non-invasive 67

Tube current measurements 65

Typical Response 20

Typographical Rules 4

- U -

Unit

Default 71

Gray 71

Prefixes 71

Röntgen 71

Units 71

Unknown device 71

Update Firmware 31

Update Mode 38, 44

Update modes 43

Update of Piranha 31

USB connector 7

Useful beam 89

User 89

- V -

View Calibrations 34

View box 69

Voxel 89

- W -

W/Rh Anode/Filter combination 54

Waste Electrical and Electronic Equipment
29

Waveform 47

Dental 63

Waveform delay 38

Waveform recording time 38, 43

Waveform recording time Specifications
19, 27

Waveform type 36, 50

WEEE 29

Weight 13

White screen 71

Whole body dose equivalent (Hw b) 89

Window 40, 41, 47

Window time 46

Workload (W) 89

- X -

Xeroradiography 90

X-ray tube 90

Notes

Notes

Notes
