



Product Note

Ocean 2014 – News in Release R16-2.7.1

Release information



Release date	2020-02-19
Release name	R16-2.7.1
Installer	2020.2A
Ocean 2014 version	2020.02.19.272
EXE file version	1.0.2.18970
DLL version	3.0.1.9131

This is a service release of Ocean 2014 for Cobia and Black Piranha.

Major change includes updated support for handling newer versions of Access Runtime Engine (up to Access 2019, v16.x).

Database format

This version of Ocean has the same database structure as last version of Ocean. Last change of database structure came with the Ocean 2014 release R15-2, 2015.12A.

Any old database from previous versions is restructured when the new version of Ocean 2014 is started. This is an irreversible process and it is not possible to go back to an earlier version once the database has been restructured. For security reasons a backup file is created automatically and saved in the Ocean 2014 data folder. The file name is "DataBackupBeforeNewVersion.accdb".

Cobia and Piranha firmware

Piranha

No new Piranha firmware version, in this release. The lowest firmware that shall be used with this release of Ocean 2014 is 4.2C.

Cobia

No new Cobia firmware version, in this release. The lowest firmware that that is recommended to be used with this version of Ocean 2014 is 2019.12A.



Product Note

Known Limitations

The table below shows issues that have been verified, and valid in this release of Ocean 2014.

Description
Under rare circumstances the Piranha may lose the Ocean license. Contact support (support@rtigroup.com) if this occurs to resolve issue.
Using Cobia and CT Dose profiler for point dose measurements in radiography, the read dose value will not be energy compensated for the added filtration set in the grid. To get energy compensated values the Total filtration value in Meter Settings has to be used.

*** END ***