



Product Note

Ocean Next – News in Release v2.0.0.1

Release information

Release date	2022-05-02
Release name	v2.0.0.1
EXE file version	1.0.2.30168
Piranha DLL (RTICmlib.dll)	3.0.1.9144
Scatter Probe DLL (rtiDeviceConnector.dll)	1.1



This is a Major Feature release of Ocean Next.

Major changes includes (but not limited to) a completely new Test View, SQLite as database instead of Microsoft Access, integration with myRTI.

Over 100 bugfixes/changes has been included in this release.

Other quality of life features includes:

- Message-panel for news and more visible calibration reminders
- Auto-start first test in sessions
- Right-click to start "new measurement"
- Touch-up in appearance in all wizards
- Improved auto-fill site functionality
- New CT calibrations
- New mammo calibrations
- New Export PDF functionality

Note that this version of Ocean installs per user. Ocean Next will only be available to the current windows user that was logged in when installing Ocean.

Connectivity

Ocean Next need to be able to communicate over the following ports: 5512 (Localhost), 5513 (Localhost), 8443 (External) to properly function.

Database format

This version of Ocean has a new database structure compared to previous versions. When installing and running this version of Ocean it will automatically update the database.

Note that the location of the Ocean database has moved to:

"C:\Users\[WindowsUser]\AppData\Local\RTI Group\Ocean Next\ProgramData\OceanData.db"



Product Note

Cobia and Piranha firmware

Piranha

This release comes with a new Firmware for Piranha. The lowest firmware that shall be used with this release of Ocean is 4.3A.

Cobia

No new Cobia firmware version, in this release. The lowest firmware that that is recommended to be used with this version of Ocean Next is 2019.12A.

RTI Scatter Probe

The latest Firmware version for RTI Scatter Probe is v1.2. There is no need to update to this version as this version is only applicable to the newest versions of the Scatter Probe hardware which already ships with this version.

Known Limitations

The table below shows issues that have been verified, and valid in this release of Ocean Next.

Description
No known limitations.

*** END ***