



INDEPENDENT X-RAY
QUALITY ASSURANCE

Use with



Mako Panoramic Holder | Specifications

Simple Positioning of Mako Dental or R/F Probes on Panoramic Dental Systems

The RTI Mako Vertical Holder allows for simple positioning of the Mako Dental or R/F PProbe onto Panoramic Dental X-ray machines, as well as vertically mounted X-ray machines.

The strap wraps around the X-ray detector to allow efficient positioning, being adjustable to tighten into place. The Holder also has a locking mechanism which allows sideways adjustment of the Probe, for fine-tuning of the Probe position.



Application



1) First connect the Mako Probe (R/F or Dental) to the Mako Panoramic Holder.



4) Place the Mako Base Unit under the strap around the back of the system for convenience (no wires drooping).



2) Connect the Mako Probe to the Mako Base Unit via USB cable.



5) Loosen the locking mechanism to make fine adjustments to the Probe position, then tighten to fix.



3) Use the strap and clip to place the Mako Probe into the correct position for X-ray exposure.

Specifications

Article number:	9765031-00
Article name:	Mako Panoramic Holder
Dimensions:	79 x 68,2 x 23,8 mm
Weight:	90 g
Material:	Aluminium
Compatible meters:	Mako Probes

